


# Communicator 5 Accessible Apps - Accessible Facebook Messenger: Quick Guide

This document provides instructions on how to use Accessible Facebook Messenger within the Communicator 5 Accessible Apps collection to stay connected with family and friends including making video calls.

**Requirements:** To use Accessible Facebook Messenger, a Facebook account is required. If a user has not logged in with Accessible Facebook previously, make sure to have Facebook login credentials available. An unlocked device with access to the Internet is also required.

**Note:** Refer to the **Getting Started with Communicator 5 Accessible Apps Quick Guide** for instructions on how to set up the Communicator 5 Home Page with the accessible apps collection.

## Launch Accessible Facebook Messenger

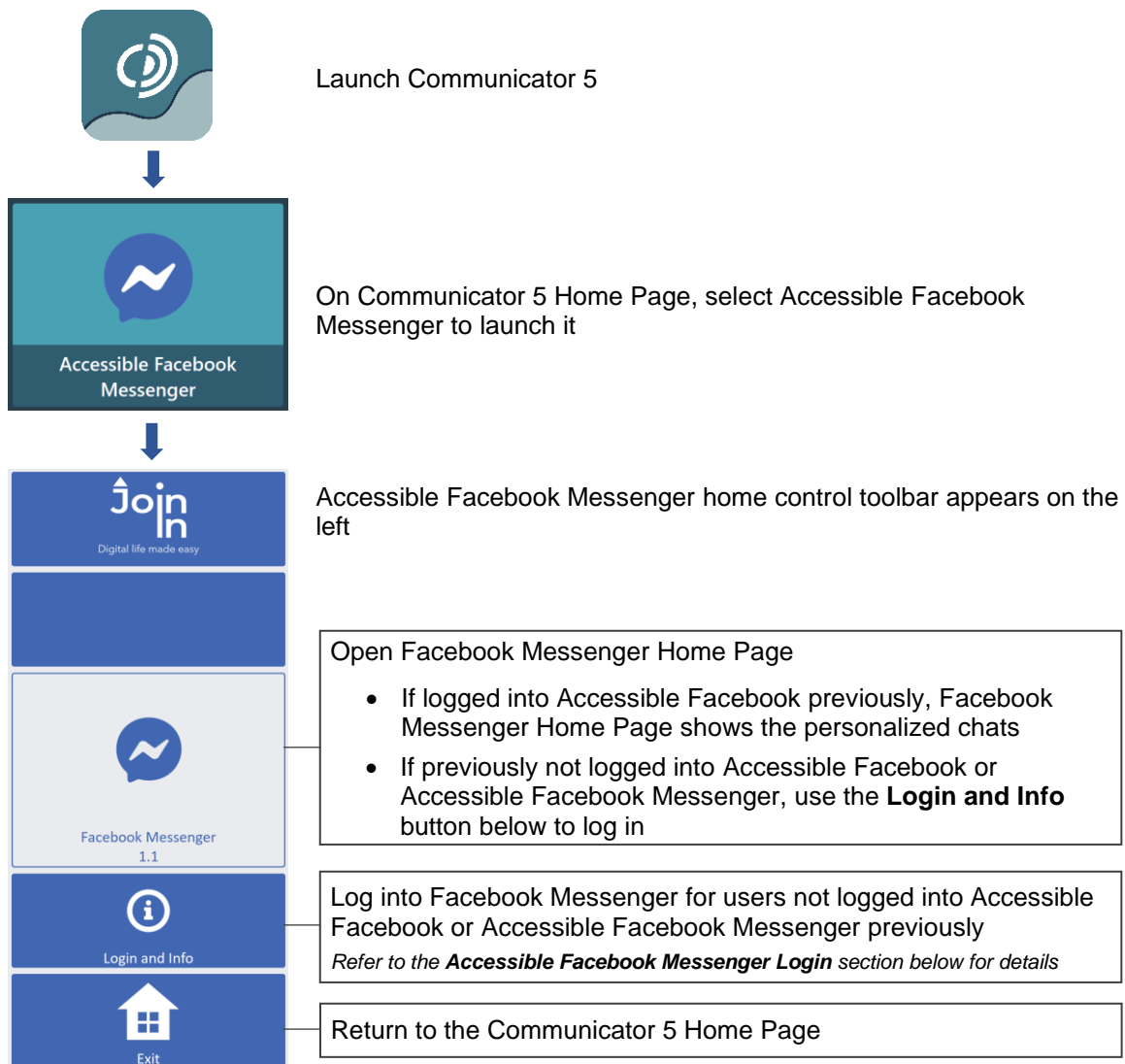

## Accessible Facebook Messenger Login

After selecting **Login and Info** on the home control toolbar, the left toolbar provides buttons for logging in, logging out and more.

Open the main control toolbar (left) and Facebook Messenger Home Page (right)

Open Accessible Facebook Messenger online training cards

Open Facebook Log In page

Log out from Facebook Messenger

Return to the Communicator 5 Home Page

facebook

Recent Logins

Create a New Account

First name Last name

Mobile number or email

New password

Birthdays

Age 21 1995

Gender Female Male Custom

Sign Up

Username and Password – bring up Communicator 5 keyboard to enter information

Log In

## Manage Contacts, Messages and Calls

After launching Accessible Facebook Messenger and login, the control toolbar provides buttons for all the functionality of Facebook Messenger in an easy and more accessible way.

Search Menu

Chat View Send Message

Previous Contact Next Contact

Browse Contacts

Messages Messages

Read Contact Call

Chats

Search Messenger

Hello how are you... 6/21/19

6/20/19, 10:50 PM

You are now connected on Messenger.

6/21/19, 1:10 AM


Hello how are you doing?

OPTIONS


Search in Conversation

PRIVACY & SUPPORT

## Main Control Toolbar

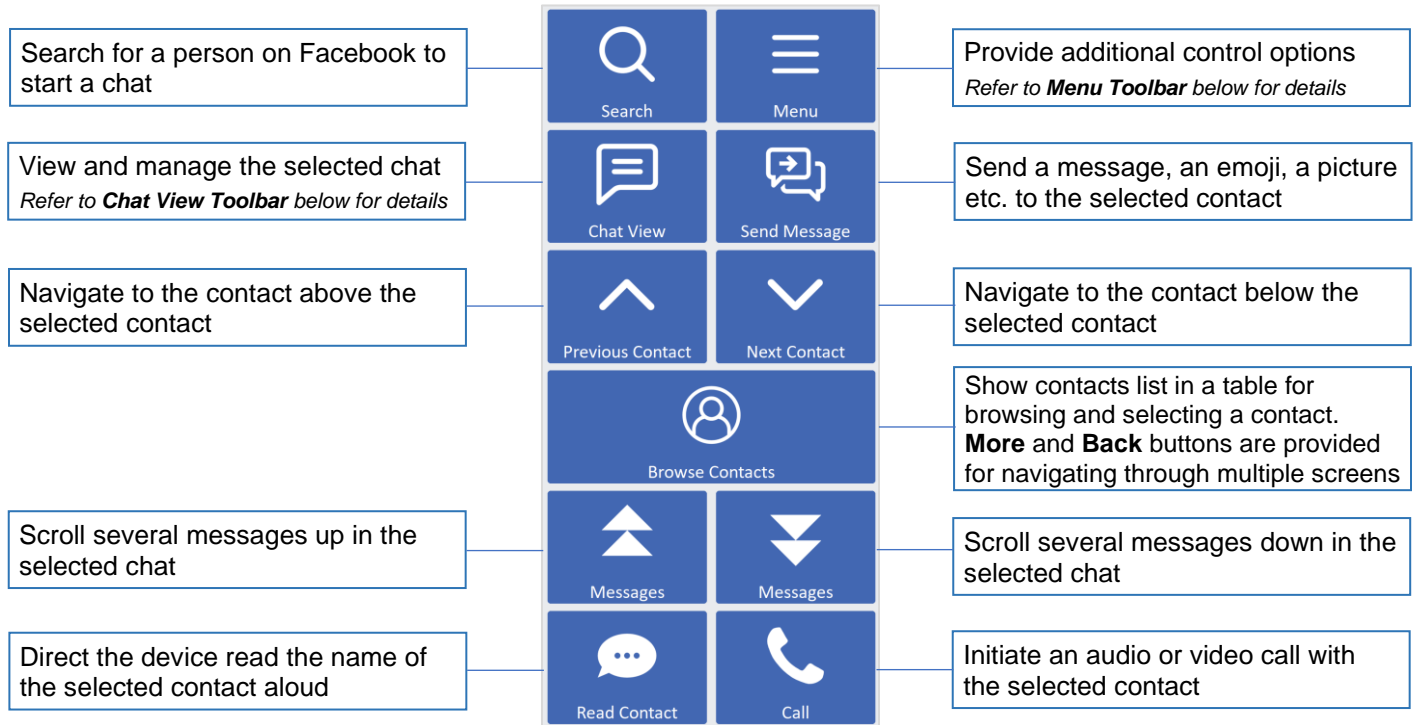

## Menu Toolbar

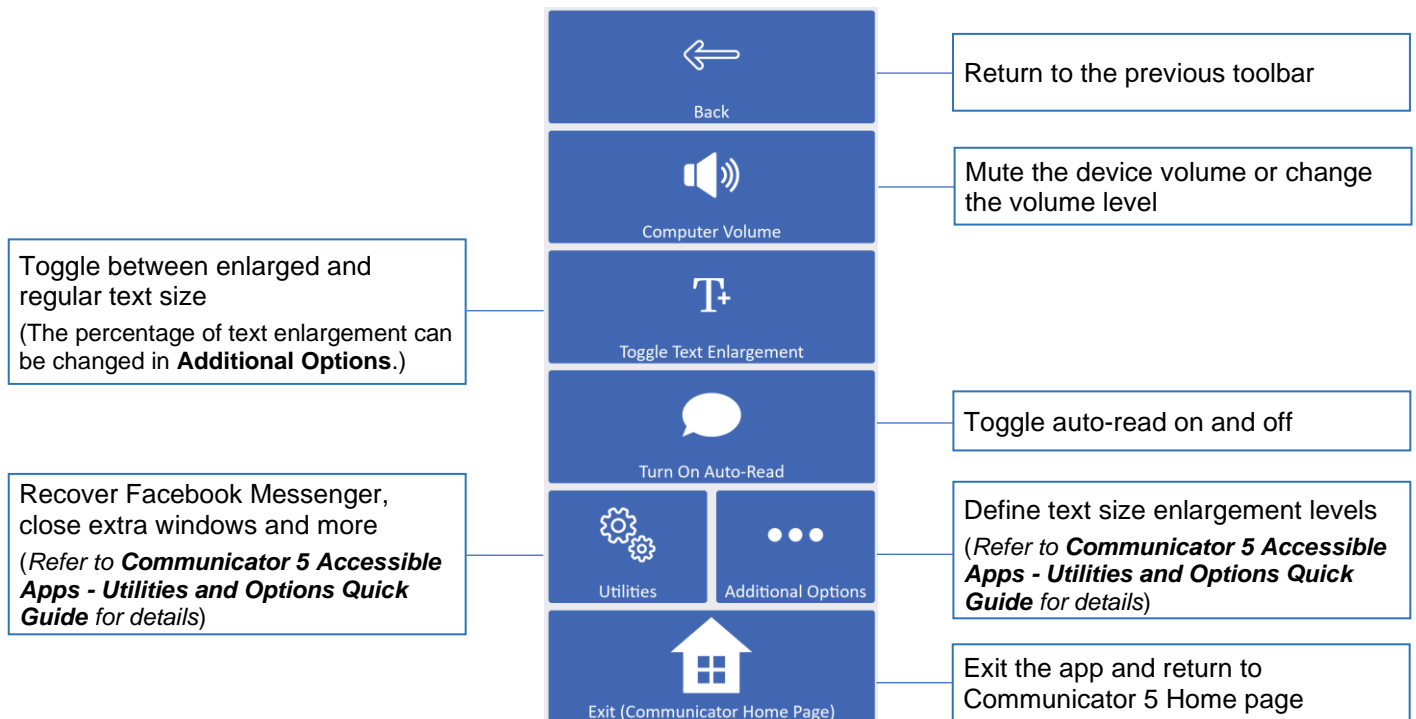

## Chat View Toolbar

