


Communicator 5

Setting up IR remote:

1. Access a **IR Remote page** from your home screen
2. To add an **IR Remote page** Open the **quick menu** (touch and hold anywhere on the screen or press this button on the left side of the device. You can also right click if you are using a mouse. Or select Ctrl + M on the keyboard)

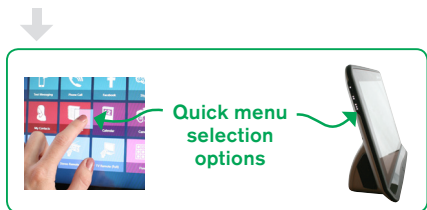

3. Select **All Page Sets**
4. Select **Text Communication**
5. Select **IR Remote Controls**
6. Choose **Simple** or **Full remote pages** for radio, dvd player and TV (Select and choose **view** to see a preview of the page)
7. Select the appropriate **remote page** that meets your needs
8. From the home page select the **IR Remote page**
9. Select the **button** you want to program (i.e Power)
10. The pop up menu will tell you that the button has no matching IR command. Select **Yes** to add a new code
11. Position remote in front of the IR receiver and follow the onscreen prompts, (you will be prompted to press and release the button a few times)
12. The button you programmed will change color if the code has successfully been learned
13. Repeat the steps until you have added all desired remote commands

For more information watch the video at

Watch dynavotech.force.com/devices/apex/Communicator

Please note you will need a mytobiidynavox.com account to access the resources

