


# Communicator 5

## Importing Communicator 4 Page Set to Communicator 5:

Communicator 5 is backwards compatible with Communicator 4, so content that has been made and used with that version can easily be imported.

**Note:** You must first backup your Communicator 4 page set before you can start this process. For step-by-step instructions, go to [www.tobii.com/en/assistive-technology/global/support-and-downloads/faqs/5013000000Sqzv/](http://www.tobii.com/en/assistive-technology/global/support-and-downloads/faqs/5013000000Sqzv/)

1. Open the **Caregiver Menu**  
(touch and hold anywhere on the screen or press this button on the left side of the device. You can also right click if you are using a mouse. Or select Ctrl +M on the keyboard)

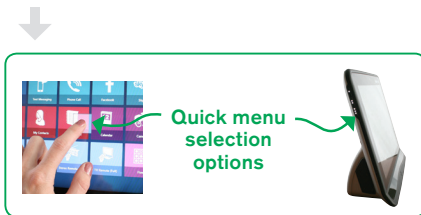

2. Select **Caregiver Settings**
3. Select the **User Settings** tab.
4. Select **(Import/Export) button**, which opens a wizard to assist you.
5. Select **Import** selected settings and data
6. Browse to the location of the files to import.
  - Select **+** next to This PC/Computer
  - Select the the location where you exported the file
  - Select the **.vse** file in the right viewport
  - Select **Next**
  - Select **Full Import.**
7. You can now decide which user to apply these Page Sets and settings to
  - An existing user will overwrite your current user and not add to the pagesets
  - A new user will create new user and not bring over any current Communicator 5 info
8. If necessary, you can later change users.