

## Release Notes **Tobii Dynavox Communicator 5**

### Introduction

Tobii Dynavox Communicator 5 is a software package designed to help individuals with communication disabilities communicate more efficiently. Communicator 5 converts text and symbols into clear speech, and offers easy-to-use tools for computer access, e-mail, text messaging, telephony, environmental control and more.

### System Requirements

- Windows 7, Windows 8, Windows 8.1 and Windows 10
- 2.0 GB RAM
- 3.0 GB Hard Drive
- 1.3 GHz processor

**Customer support:** For more information or support, please visit [www.TobiiDynavox.com](http://www.TobiiDynavox.com).

### Release Version 5.4.2

**Release date:** 2017-12-19

**Release languages:** All communicator languages

**Release channel:**

- Tobii Dynavox Web
- USB stick
- Check for UpdatesTobii Dynavox Update Notifier

### New Features

- Changed buttons size and default settings for Scanning Mouse Emulation (TDCOM-854)
- Ability to run Tobii Dynavox applications from a button action in Communicator (TDCOM-801)
- Default input method on touch devices is "Touch/Mouse" now (TDCOM-778)
- Reduce number of activations in Chinese to 1 per license (TDCOM-845)

### Improvements

- Fixed many Windows Onscreen keyboard related issues (TDCOM-930)
- Fix for SonoKey users who couldn't use switch based mouse emulation (TDCOM-715)
- Tobii Dynavox Device Switch Port as default on I-110 (CVI-1143)
- The verb "be" is incorrect in Grammar tool (TDCOM-682)
- Windows Control On/Off button in SonoKey does not appear on devices that do not have IS4 eye trackers (TDCOM-718)
- Updating to Communicator 5.4 makes eye-tracking settings appear empty (TDCOM-776)
- Communicator crash when changing eye-tracking activation method (TDCOM-772)
- Japanese not translated strings (CVI-1520)
- IR learns signals from eye tracker on EyeMobile+ (TDCOM-730)
- Copyright on tablet doesn't look nice (TDCOM-769)
- "Calibration is needed" warning is not noticeable enough (CVI-1541)
- Using Scanning in LiterAACyPro 10x7 causes an "Object reference..." error (TDCOM-727)
- Companion License not displayed on I-110 (TDCOM-742)

### Support for Your Tobii Dynavox Device

#### Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: [www.TobiiDynavox.com](http://www.TobiiDynavox.com) or [www.myTobiiDynavox.com](http://www.myTobiiDynavox.com).

#### Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit [www.TobiiDynavox.com/contact](http://www.TobiiDynavox.com/contact)

- Firefox Page Set in C5 is not compatible with German Firefox (TDCOM-620)
- Can't untick "Display a preview of incoming messages" (TDCOM-779)
- Server Busy Error when creating new user when having TET user profile with different casing (TDCOM-817)
- Grammar problems in LiterAACy: You/they/we am (CVI-1013)
- I-Series switch setup is not suggested on I-Series+ (TDCOM-799)
- Wrong tenses of "can" in Sono Scribe (CVI-1011)
- Snap + Core First opens minimized when using Snap button on Communicator Home Page (CVI-851)
- Crash in C5 while changing language and prediction (TDCOM-843)
- Music Player. Looks like only one music file is shown (TDCOM-819)
- Crash at startup in MSVCR120.dll (TDCOM-806)
- Server busy error on new user creation (TDCOM-925)
- Can't input Chinese characters in Chinese C5 (Edit Button) from Indi (TSAR-391)



This was fixed by Microsoft in Windows RedStone 3. So, users have to update Windows to see a fix

## Release Version 5.4.1

**Release date:** 2017-09-07

**Release languages:** All communicator languages

**Release channel:**

- Tobii Dynavox Web
- Tobii Dynavox I-110 deploy

### New Features

- Support for Tobii Dynavox I-110 hardware services (CVI-1143, CVI-1140)
- Auto licensing on Tobii Dynavox I-110 device (CVI-1130)

## Release Version 5.4

**Release date:** 2017-06-20

**Release languages:** All communicator languages

**Release channel:**

- Tobii Dynavox Web
- USB stick
- Check for Updates
- Tobii Dynavox Update Notifier

### New Features

- UI changes in Keyboards and Quick Talk (CVI-1344)
- Copy & Delete multiple pages (CVI-1164)
- Hide/Unhide buttons in Edit Button Dialog (CVI-1239)
- Change font size on all page set buttons (CVI-1146)
- Launch Tobii Dynavox applications from Communicator 5 (CVI-797)
- Speak with recorded messages (CVI-1238)
- Improved performance on opening page sets (CVI-1234)

## Support for Your Tobii Dynavox Device

### Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: [www.TobiiDynavox.com](http://www.TobiiDynavox.com) or [www.myTobiiDynavox.com](http://www.myTobiiDynavox.com).

### Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit [www.TobiiDynavox.com/contact](http://www.TobiiDynavox.com/contact)

- Smaller suite installer in .exe file (CVI-1242)
- Support for receiving MMS messages when using the Beam app (SCHN-1371)
- Compatibility: Communicator is running with PCEye Plus, Eye Mobile Plus, On I-110, Indi device on all languages

## Improvements

- As a user, I want to set speak this text using content tool quicker (TDCOM-577)
- Renaming a PageSet in Organize My Page Sets does not change its name on the Home Page (TDCOM-450)
- After e-mail password has changed editing the email settings does not seem to work (CVI-1246)
- There is a big lag when using the content edit tool in C5 (TDCOM-639)
- 'Beam Beta Information' and 'Telephony not available' buttons are not visible on Indi because Device Visibility button works incorrectly (TDCOM-652)
- Numerous small bug fixes

## Release Version 5.3.2

**Release date:** 2017-04-24

**Release languages:** en-US, en-UK, de-DE, zh-CN and ja-JP

**Release channel:**

- On Indi™ — en-US, en-UK, de-DE and zh-CN
- On Update Notifier and Check for Updates — zh-CN & ja-JP

## New Features

- Communicator uses the latest version of IR
- The HBG IR Remote control is automatically selected on a Indi™ device
- On a licensed Indi™ device Communicator has a gold license and recognizes that it's on Indi™ device

## Improvements

- Not possible to save any .cddx file on an external drive (except C:) (TDCOM-565)
- Page Sets cannot be saved when opened from removable device (TDCOM-451)
- Impossible to update Communicator through Check for Updates (CVI-1184)
- (Chinese) IME suggestions do not appear while editing (CVI-1097)
- Help and Training links open tobiidynavox start page (CVI-994)
- Moving from button to button lags or takes longer in 5.3.1 than in previous versions (TDCOM-610)
- (Japanese) IME is turned off automatically (TDCOM-618)
- (Japanese) Incorrect predictions are chosen from word prediction pop-up window (TDCOM-624)
- Problem with IME Japanese input (TDCOM-576)

## Release Version 5.3.1

**Release date:** 2016-12-09

## Improvements

- Saving page sets to a non C Drive location failed (TDCOM-565)
- Possible to get trapped in Content Products if set as Startup Page (TDCOM-566)

## Support for Your Tobii Dynavox Device

### Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: [www.TobiiDynavox.com](http://www.TobiiDynavox.com) or [www.myTobiiDynavox.com](http://www.myTobiiDynavox.com).

### Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit [www.TobiiDynavox.com/contact](http://www.TobiiDynavox.com/contact)

## Release Version 5.3

**Release date:** 2016-11-30

### New Features

- You can now add and launch Files from the Home Page.
- The Voice Pitch can be adjusted for Acapela Voices, which allows you to adjust the age of children's voices.
- Scanning Windows Control had been updated to provide better contrast on all backgrounds.
- Communicator can now import messages from the online Message Banking Service.
- You can use Beam to send SMS on Android phones on any Windows 10 PC.
- New Keyboard Layouts and a new High-Contrast color scheme.
- Edit Multiple buttons in Run view (changes to Edit button dialogue)
- Quick talk functionality
- Presentation mode in My Text Files page set

### Improvements

- Licensing was not functioning when the device was offline (TSAR-217)
- Improved saving times (CVI-647)
- Fixed bugs related to losing focus in Windows 10 Tablet mode (CVI-589)

## Release Version 5.2.2

**Release date:** 2016-10-31

### Improvements

- In Sono Flex, moving between pages will cause the message window to speak (TDCOM-537)



Only for English, Danish and German

## Release Version 5.2.1

**Release date:** 2016-08-29

### Improvements

- In Sono Flex, when 2 words on a label, speak as you type sound setting does not work. (TDCOM-515)



Only for English, Danish and German

## Release Version 5.2

**Release date:** 2016-04-25

### Support for Your Tobii Dynavox Device

#### Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: [www.TobiiDynavox.com](http://www.TobiiDynavox.com) or [www.myTobiiDynavox.com](http://www.myTobiiDynavox.com).

#### Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit [www.TobiiDynavox.com/contact](http://www.TobiiDynavox.com/contact)

## New Features

- Change the Dwell Free Keyboard from a Beta Feature to a full Communicator Keyboard
- Dwell-Free has been translated into German
- Dwell-Free can be set as a Communicator Keyboard
- The Dwell-Free prediction engine will learn new words
- The error correction mode in Dwell-Free provides Swiftkey word predictions as you type
- Dwell-Free has been improved to allow for long text editing, with improved cursor movement options and the ability to insert new sentences to the location of the cursor
- Dwell-Free now has auto-correct features, such as intelligent spacing for punctuation and auto-capitalization

## Improvements

- Crash when duplicating non-button objects (CVI-548)
- Japanese Window Keyboards cannot turn IME on/off (CVI-484)
- Japanese Large Keys Keyboard displays incorrect second page for “ふ” (CVI-483)
- Italian abc Keyboard layout is missing àùè (CVI-475)
- Activating Communicator Fails if the Machine Name contains non-English characters (CVI-434)
- In Sono Key navigating to the Alarm page clears the Message Window (CVI-411)

## Release Version 5.1

**Release date:** 2015-12-09

## New Features

- The new Abbreviation Expansion feature allows users to access their phrases faster than ever.
- There are new Alarm and Go To Desktop page sets
- The user can set a Different voice to be used for speech output and feedback sounds
- The Combine 2 IR commands action has been updated to allow the user to also Combine 3 IR commands
- Added Keyboard support for Croatian, Czech, Hungarian, Polish, Slovak, Slovenian, & Ukrainian
- Communicator 5 is now available in Finnish, Brazilian Portuguese, & Japanese

## Improvements

- Communicator leaks memory when loading cddx files (TDCOM-327)
- Links refer to [www.pagesetcentral.com](http://www.pagesetcentral.com) (TDCOM-311)
- Importing TC4 E-Mail into C5 may cause a crash on startup (TDCOM-256)
- Unexpected results when injected page has the same name as a page in the main page set (TDCOM-193)
- Skype OSK Page Set cannot place calls (TDCOM-294)

## Release Version 5.0.1

**Release date:** 2015-09-22

## Improvements

- Improved reliability when sending or receiving text messages through a Bluetooth phone (CFIVE-6573)
- Updated Facebook to use the new API 2.0 (CVI-17)
- Occasionally unable to Save Communicator 4 Page Set in Communicator 5 (TDCOM-258)

## Support for Your Tobii Dynavox Device

### Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: [www.TobiiDynavox.com](http://www.TobiiDynavox.com) or [www.myTobiiDynavox.com](http://www.myTobiiDynavox.com).

### Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit [www.TobiiDynavox.com/contact](http://www.TobiiDynavox.com/contact)