

## Release Notes **Tobii Communicator 4**

### Introduction

Tobii Dynavox Communicator 4 is a software package designed to help individuals with communication disabilities communicate more efficiently. Communicator 4 converts text and symbols into clear speech, and offers easy-to-use tools for computer access, e-mail, text messaging, telephony, environmental control and more.

### System Requirements

- Windows XP, Vista (SP2), 7, 8 or 8.1
- .NET Framework 4 Full

**Customer support:** For more information or support, please visit [www.TobiiDynavox.com](http://www.TobiiDynavox.com).

### Release Version 4.8.9

**Release date:** 2015-10-23

#### Improvements

- Update Facebook API to v2 (VSCOM-4743)
- Skype 7.0 not working with our new page set (VSCOM-4763)
- If Communicator 4 and Communicator 5 is licensed at the same time, TC4 starts in Evaluation mode (VSCOM-4770)
- Communicator 4 does not recognize I-Series features if Communicator 5 flag is in I-Series product key (VSCOM-4795)
- Not possible to use images with jpeg extension (VSCOM-4048)
- Bluetooth: Automatically sending of messages in outbox when a phone is connected/reconnects does not work if inbox on the phone is empty (VSCOM-4254)
- Sending and receiving of SMS not possible on Xperia Z (VSCOM-4715)
- "Do no keep history of recently opened documents" policy in Windows crashes TC at startup (VSCOM-4765)
- Music Player does not always show album art, even when Microsoft Media Player has downloaded the album art (VSCOM-140)
- TC sometimes fails to connect using MAP profile if there are no messages on the phone (Sony XPERIA Z) (VSCOM-4473)
- Translation/misspelling in German TC (VSCOM-4609)
- "Windows Control" page set needs to be run by administrator (VSCOM-4766)
- No notification message if Facebook page would be opened in Sono Key without setup account settings (VSCOM-4779)
- In Complete Text Messaging the list of phrases cannot be scrolled (VSCOM-4788)
- Messages in Sent Items disappear on restarting Communicator (VSCOM-4789)

### Release Version 4.8.8

**Release date:** 2014-09-01

#### Improvements

- Communicator crashes when it tries to log in to Skype (VSCOM-4729)
- C8: content products in evaluation mode with TC 4.8.7 (VSCOM-4716)
- Any license key with Medicare or Compass enabled leaves Communicator unlicensed (TAM-1814)

### Support for Your Tobii Dynavox Device

#### Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: [www.TobiiDynavox.com](http://www.TobiiDynavox.com) or [www.myTobiiDynavox.com](http://www.myTobiiDynavox.com).

#### Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit [www.TobiiDynavox.com/contact](http://www.TobiiDynavox.com/contact)

## Release Version 4.8.7

**Release date:** 2014-06-30

### New Features

- "Leave button before reselect" - when eye tracking is used as input method, it is possible to prevent clicks on a newly opened page.

### Improvements

- Wrong texts in Japanese GazeInteractionSettings.cdd (VSCOM-4693)
- BT: Mobile phone Signal strength is not updated (VSCOM-4665)
- TC crashes on first button selection after start up if input method is scanning or dwell (VSCOM-4657)
- ModelTalker voices not working in Tobii Communicator (VSCOM-4645)
- SkypeKit tries to sign in to Skype on unsupported Windows 8.1 OS (VSCOM-4642)
- Blackberry Z10 - is sending message but it is placed in outbox and reported as not sent (VSCOM-4640)
- Chinese texts in Japanese Suite installer (VSCOM-4639)
- Words in Wrong Word Class in German Grammar Database (VSCOM-4633)
- Eye tracker feedback dot should be hidden while playing sounds with "Prevent user Interaction..." selected (VSCOM-4620)
- The default size of the Smart Button Type and Actions dialog box should be bigger by default (VSCOM-4612)
- SMS longer than 1024 character in paired phone's inbox crashes the Communicator (VSCOM-4607)
- Phone connects as Calling Only (VSCOM-4574)
- German Communicator does not contain all inflections of pronouns (VSCOM-4530)
- SK-796: Scanning does not work on I-Series devices with Sono Key (new scan setting for I-Series not supported) (VSCOM-4435)
- When updating to 4.8.1 IR signals did not transfer over. (VSCOM-4385)
- Green Camera Feed on Dell Tablets (VSCOM-4187)
- About 200 Misspelled Words in German Grammar Database (VSCOM-4142)
- Sono Key not implicitly licensed because C-Series HWO doesn't start fast enough (VSCOM-4061)
- Implement "Leave button before reselect" when eye tracking (VSCOM-3870)

## Release Version 4.8.6

**Release date:** 2014-03-18

### Improvements

- Implement "Leave button before reselect" when eye tracking (VSCOM-3870)
- Sono Key not implicitly licensed because C-Series HWO doesn't start fast enough (VSCOM-4061)
- Words in Wrong Word Class in German Grammar Database (VSCOM-4633)
- About 200 Misspelled Words in German Grammar Database (VSCOM-4142)
- Scanning does not work on I-Series devices with Sono Key (new scan setting for I-Series not supported) (VSCOM-4435)
- SMS longer than 1024 character in paired phone's inbox crashes the Communicator (VSCOM-4607)
- Crash when 'Leave button before reselect' is on (VSCOM-4658)
- TC crashes on first button selection after start up if input method is scanning or dwell (VSCOM-4657)
- Chinese texts in Japanese Suite installer (VSCOM-4639)
- German Communicator does not contain all inflections of pronouns (VSCOM-4530)
- Translation/misspelling in German TC (VSCOM-4609)

### Support for Your Tobii Dynavox Device

#### Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: [www.TobiiDynavox.com](http://www.TobiiDynavox.com) or [www.myTobiiDynavox.com](http://www.myTobiiDynavox.com).

#### Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit [www.TobiiDynavox.com/contact](http://www.TobiiDynavox.com/contact)

## Release Version 4.8.5

**Release date:** 2014-03-25



4.8.5 is made especially for users of M-Series units.

### Improvements

- Licensing Feature for M-Series (VSCOM-4520)
- Green Camera Feed on Dell Tablets (VSCOM-4187)



This version is only deployed on M-Series. It is not available anywhere else.

## Release Version 4.8.4

**Release date:** 2014-01-14

### Improvements

- Installer for bilingual American-Mexican Children's Voices (VSCOM-4488)
- Opening Tool > My Recorded Sounds after import crashes TC (VSCOM-4475)
- E-mails that are deleted from the deleted folder, will be downloaded again next time user presses Send/Receive (VSCOM-4446)
- "File is encrypted or is not a database" message on startup (VSCOM-4519)
- In email settings there is no longer a way to repair or purge email folders. (VSCOM-4294)
- When scanning and using Audible Cue the first item in the group is not spoken (no cue) the first time (VSCOM-4430)
- SK-796: Scanning does not work on I-Series devices with Sono Key (new scan setting for I-Series not supported) (VSCOM-4435)
- Scanning Input Settings changes from Joystick to Keyboard without warning (VSCOM-4392)
- 'Manage Communicator Window' - 'Toggle Window Style' works differently than F10 (VSCOM-4278)
- BT: Communicator hangs/crashes after hibernate if the connected phone doesn't accept access to SMS messages (VSCOM-4248)
- A Few Acapela Voices No Longer Shipped with Communicator (VSCOM-4349)
- Logger should log Errors by default (VSCOM-4543)
- Disconnecting IR modules causes TC to spike in CPU usage (VSCOM-4456)
- Import problem due to absolute linking (VSCOM-3767)
- If user select to not import My Recorded sounds when importing, the My Recorder sounds library path will be wrong after import. (VSCOM-1582)
- Possibility to refer to page sets located in "system variable folder" (VSCOM-4560)
- Page Set Not Found error when exporting from C-Series and importing to I-Series (VSCOM-4454)
- Smart button type "Month and Year" does not update using "Select previous year" or "Select next year" (VSCOM-4439)
- Category scroll buttons on Regular.cdd do not scroll (VSCOM-4504)
- Mobile phone settings, Swedish translation (VSCOM-4344)
- Make it possible to define the volume for Acapela 8 and 9 voices by setting a value in the registry (VSCOM-4564)
- Wrong default action on TV Remote (Full) (VSCOM-4561)
- German grammar database contains several words with \$ instead of ß (VSCOM-4556)
- User interface for enabling logs (VSCOM-4555)
- Modify start end/date for appointment action (VSCOM-4500)
- E-mail attachments are not deleted from disk when Deleted Items folder is emptied (Delete All) (VSCOM-4499)
- Update Next/prev month action (VSCOM-4442)

## Support for Your Tobii Dynavox Device

### Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: [www.TobiiDynavox.com](http://www.TobiiDynavox.com) or [www.myTobiiDynavox.com](http://www.myTobiiDynavox.com).

### Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit [www.TobiiDynavox.com/contact](http://www.TobiiDynavox.com/contact)

- As a caregiver I want to be able to import Extended Phrase list in US English Communicator (VSCOM-4529)
- German Grammar database - "':" shows on some Akkusativ button (VSCOM-4059)

## Release Version 4.8.3

**Release date:** 2013-09-30

### Improvements

- TC is not able to send and download e-mails with inbox.com account (VSCOM-4444)
- "Server for outgoing e-mails (SMTP) requires authorization" should be selected by default (VSCOM-4443)
- Pop3 error 64 certificate invalid or not trusted (VSCOM-4434)
- TC Message boxes are not eye trackable (VSCOM-4431)
- Script error when attempting to log into Facebook (VSCOM-4418)
- Pop3 Error 11 could not resolve host (VSCOM-4412)
- Sound setting dialogue not optimized for lower screen resolution (VSCOM-4405)
- User not able to send e-mails after upgrading to TC 4.8.1 (Unterminated line sent by server) (VSCOM-4404)

## Release Version 4.8.2

**Release date:** 2013-07-18

### Improvements

- "Send and receive" will not work if casing of pop or smtp server name is wrong (VSCOM-4361)
- Opening a page set file when a popup is open causes crash (VSCOM-4081)
- If you have the same word twice in the alternative pronunciation map, Communicator will crash at startup (VSCOM-4352)
- Using e-mail page set while Communicator downloads new e-mails may result in Communicator crash (VSCOM-4333)
- Pressing Send/Receive while emails are automatically downloaded by Communicator in background gives an error (VSCOM-4366)
- Communicator 4 will crash if you open a Communicator 3 page set (VSCOM-4370)
- Many log entries "InvokeHelper is not initialized" (VSCOM-4373)
- "Copy Word in Form to Variable" action does not work (VSCOM-4372)
- Communicator is not responding after sleep or hibernate (VSCOM-4340)
- TC fails to import some emails from old email store to new DB (VSCOM-4387)
- After importing old mail store to new DB, restoring a deleted email does not work (works for emails deleted after import) (VSCOM-4388)
- To/From information is missing from some e-mail after importing from old mailstore (VSCOM-4386)
- IR commands do not work after updating from 4.7 to 4.8 (VSCOM-4368)
- Your word prediction dictionary was corrupted error message looping (VSCOM-4382)

## Support for Your Tobii Dynavox Device

### Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: [www.TobiiDynavox.com](http://www.TobiiDynavox.com) or [www.myTobiiDynavox.com](http://www.myTobiiDynavox.com).

### Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit [www.TobiiDynavox.com/contact](http://www.TobiiDynavox.com/contact)