Safety and Compliance EyeMobile Plus by Tobii Dynavox

1 Safety

The EyeMobile Plus device has been tested and approved as compliant to all the Specifications and Standards listed in 2 *Compliance Information, page 2*. Nevertheless, in order to ensure safe operation of your EyeMobile Plus device, there are a few safety warnings to bear in mind:

No modification of this equipment is allowed

The EyeMobile Plus is not to be used as a life supporting Device, and it shall not be relied on in case of loss of function due to power loss or other causes.

There could be a choking hazard risk if small parts detach from the EyeMobile Plus device.

The EyeMobile Plus device shall not be exposed to or used in rain or weather conditions outside the Technical Specification of the EyeMobile Plus device.

Young children or people with cognitive disabilities should not have access to, or the use of, the EyeMobile Plus device without parental or guardian supervision.

The EyeMobile Plus device shall be used with care when moving around.

1.1 Avoiding Hearing Damage

Permanent hearing loss may occur if earphones, headphones or speakers are used at high volume. To prevent this, the volume should be set to a safe level. You can become desensitized over time to high sound levels which may then sound acceptable yet still could be damaging your hearing. If you experience symptoms such as ringing in your ears, please lower the volume or stop using the earphones/headphones. The louder the volume, the less time is required before your hearing could be affected.

Hearing experts suggest the following measures to protect your hearing:

- Limit the amount of time you use earphones or headphones at high volume.
- Avoid turning up the volume to block out noisy surroundings.
- Turn the volume down if you cannot hear people speaking near you.

To establish a safe volume level:

- Set your volume control at a low setting.
- Slowly increase the sound until you can hear it comfortably and clearly, without distortion.

1.2 Power Supply and Batteries

The EyeMobile Plus device contains a rechargeable battery. All rechargeable batteries degrade over time. Thus the possible usage times for the EyeMobile Plus after a full charge can become shorter over time than when the device was new.

The EyeMobile Plus device uses a Li-ion Polymer battery.

If you are in a hot environment, be aware that it can affect the ability to charge the battery. The internal temperature has to be between 0 °C/32 °F and 45 °C/113 °F for the battery to charge. If the internal battery temperature raises above 45 °C/113 °F the battery will not charge at all.

Move the EyeMobile Plus device to a cooler environment to let the battery charge properly.

Avoid exposing the EyeMobile Plus device to fire or to temperatures above 60 °C/140 °F. These conditions may cause the battery to malfunction, generate heat, ignite or explode. Be aware that it is possible, in a worst case scenario, for temperatures to reach greater than those stated above in, for example, the trunk of a car on a hot day. So, storing the EyeMobile Plus device, in a hot car trunk could conceivably lead to a malfunction.

Only charge the EyeMobile Plus battery in an ambient temperature of 0 °C/32 °F to 45 °C/113 °F.

Use only the supplied power adapter to charge the EyeMobile Plus device. Using unauthorized power adapters may severely damage the EyeMobile Plus device.

For safe operation of the EyeMobile Plus device, use only charger and accessories approved by Tobii AB.

Do not open, or modify, the casing of the EyeMobile Plus device or of the power supply, since you may be exposed to potentially hazardous electrical voltage. The device contains no serviceable parts. If the EyeMobile Plus device or its accessories are mechanically damaged, **do not to use them**.

If the battery is not charged and the EyeMobile Plus is not connected to the power supply, the EyeMobile Plus device will shut down.

If the Power Supply Cord is damaged it needs to be replaced by Service Personnel only. Do not use the Power Supply Cord until replaced.

Special regulations apply to shipping devices containing Li-ion Polymer batteries. If dropped, crushed, or short-circuited, these batteries can release dangerous amounts of heat and may ignite, and are dangerous in fires.

Please reference IATA regulations when shipping lithium metal or lithium ion batteries or cells: http://www.iata.org/whatwedo/cargo/dangerous_goods/Pages/lithium_batteries.aspx

The Power adapters shall not be used without an adult or Caregiver supervision.

1.3 Mounting

The EyeMobile Plus should be mounted according to the instructions of the approved mounts used. Tobii AB or its agents are not liable for damage or injuries to a person or its property due to a EyeMobile Plus falling from a mounted configuration. The mounting of a EyeMobile Plus is done entirely at the user's own risk.

1.4 Epilepsy Warning

Some people with **Photosensitive Epilepsy** are susceptible to epileptic seizures or loss of consciousness when exposed to certain flashing lights or light patterns in everyday life. This may happen even if the person has no medical history of epilepsy or has never had any epileptic seizures.

A person with Photosensitive Epilepsy would also be likely to have problems with TV screens, some arcade games, and flickering fluorescent bulbs. Such people may have a seizure while watching certain images or patterns on a monitor, or even when exposed to the light sources of an eye tracker. It is estimated that about 3-5% of people with epilepsy have this type of Photosensitive Epilepsy. Many people with Photosensitive Epilepsy experience an "aura" or feel odd sensations before the seizure occurs. If you feel odd during use, move your eyes away from the eye tracker.

1.5 Infrared Warning

When activated, the EyeMobile Plus emits pulsed infrared (IR) light. Certain medical devices are susceptible to disturbance by IR light and/or radiation. Do not use the EyeMobile Plus when in the vicinity of such susceptible medical devices as their accuracy or proper functionality could be inhibited.

1.6 Magnetic Field Warning

The EyeMobile Plus Magnetic Mounting contains magnets. Magnetic fields may interfere with the function of cardiac pacemakers and implantable cardioverter-defibrillators. As a general rule, maintain a minimum distance of 6 inches (15 centimeters) between any item with magnets and your heart device.

1.7 Child Safety

The EyeMobile Plus is an advanced computer system and electronic device. As such it is composed of numerous separate, assembled parts. In the hands of a child certain of these parts have the possibility of being separated from the device, possibly constituting a choking hazard or another danger to the child.

Young children should not have access to, or use, the device without parental or guardian supervision.

1.8 Do not open EyeMobile Plus

Non-compliance will result in loss of Warranty! There are no user serviceable components inside. Contact Tobii Dynavox Support if your EyeMobile Plus is not working properly.

1.9 Emergency

Do not rely on the device for emergency calls or banking transactions. We recommend having multiple ways to communicate in emergency situations. Banking transactions should only be carried out with a system recommended by, and approved according to the standards of, your bank.

1.10 Windows Control

Some people may experience a certain amount of fatigue (due to intentional eye focusing and hard concentration) or even a dryness of the eyes (due to less frequent blinking) when first getting used to Windows Control. If you are experiencing fatigue or dry eyes start off slowly and limit the length of your Windows Control sessions to your comfort level. Remoisturizing eye drops can be helpful to combat dryness.

1.11 Third Party

Any use of the EyeMobile Plus outside the intended use and together with any third party software or hardware that changes the intended use is a risk and nothing that Tobii AB can take any responsibility for.

1.12 Environmental Control

The Programmable IR Remote Control — Environmental Control Unit (ECU) is not to be relied on as the only method of interaction with IR controlled devices.

2 Compliance Information

The EyeMobile Plus is CE-marked, indicating compliance with the essential health and safety requirements set out in European Directives..

2.1 FCC Statement

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Modifications not expressly approved by Tobii AB could void the user's authority to operate the equipment under FCC rules.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications.

However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into a wall outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or experienced radio/TV technician for assistance.

2.2 Industry Canada Statement

This Class B digital apparatus complies with Canadian ICES-003.

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2.3 CE Statement

The EyeMobile Plus complies with the following Directives:

- 2014/30/EU (EMC) - Electromagnetic Compatibility Directive
- 2014/35/EU (LVD) - Low Voltage Directive
- 2011/65/EU (RoHS) - Restriction of Hazardous Substances Directive
- 2012/19/EU - WEEE Directive
- 2006/121/EC, 1907/2006/EC Annex 17 - Reach Directive

2.4 Standards

The EyeMobile Plus complies with the following standards:

- FCC part 15, Class B
- ICES-003 Issue 6:2016 Class B
- EN 55032: 2012+AC: 2013 Class B
- EN 55024:2010+A1:2015
- EN 61000-3-2: 2014
- EN 61000-3-3: 2013
- IEC 61000-4-2 Ed. 2.0: 2008
- IEC 61000-4-3 Ed. 3.2: 2010
- IEC 61000-4-4 Ed. 3.0: 2012
- IEC 61000-4-5 Ed. 3.0: 2014
- IEC 61000-4-6 Ed. 4.0: 2013
- IEC 61000-4-8 Ed. 2.0: 2009
- IEC 61000-4-11 Ed. 2.0: 2004

- FCC CFR Title 47 Part 15 Subpart B: 2015, Class B
- CISPR 22: 2008, ANSI C63.4: 2014
- AS/NZS CISPR 32: 2013, Class B

3 Customer Support

For support, please contact your local representative or Tobii Dynavox Support. In order to receive assistance as quickly as possible, make sure you have access to your EyeMobile Plus and, if possible, an Internet connection. You should also be able to supply the serial number of the device, which you find on the back of the device. Find our Support pages online at: www.tobiidynavox.com or www.myTobiiDynavox.com.

4 Disposing of the EyeMobile Plus

Do not dispose of the EyeMobile Plus in general household or office waste. Follow your local regulations for the disposal of electrical and electronic equipment.

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