Safety and Compliance

1 Safety

The Tobii Dynavox Indi™ device has been tested and approved as compliant to all the Specifications and Standards listed in 2 *Compliance Information*, page 2 of this manual and in the 5 *Technical Specifications*, page 2. Nevertheless, in order to ensure safe operation of your Tobii Dynavox Indi™ device, there are a few safety warnings to bear in mind:

No modification of this equipment is allowed

The user may never attempt to change the battery. Altering of the battery may cause risk of explosion.

The Tobii Dynavox Indi™ is not to be used as a life supporting Device, and it shall not be relied on in case of loss of function due to power loss or other causes.

There could be a choking hazard risk if small parts detach from the Tobii Dynavox Indi™ device.

The Tobii Dynavox Indi™ device shall not be exposed to or used in rain or weather conditions outside the Technical Specification of the Tobii Dynavox Indi™ device.

Young children or people with cognitive disabilities should not have access to, or the use of, the Tobii Dynavox Indi™ device without parental or guardian supervision.

The Tobii Dynavox Indi™ device shall be used with care when moving around.

1.1 Avoiding Hearing Damage

Permanent hearing loss may occur if earphones, headphones or speakers are used at high volume. To prevent this, the volume should be set to a safe level. You can become desensitized over time to high sound levels which may then sound acceptable yet still could be damaging your hearing. If you experience symptoms such as ringing in your ears, please lower the volume or stop using the earphones/headphones. The louder the volume, the less time is required before your hearing could be affected.

Hearing experts suggest the following measures to protect your hearing:

- Limit the amount of time you use earphones or headphones at high volume.
- Avoid turning up the volume to block out noisy surroundings.
- Turn the volume down if you cannot hear people speaking near you.

To establish a safe volume level:

- Set your volume control at a low setting.
- Slowly increase the sound until you can hear it comfortably and clearly, without distortion.

1.2 Power Supply and Batteries

The Tobii Dynavox Indi™ device contains a rechargeable battery. All rechargeable batteries degrade over time. Thus the possible usage times for the Tobii Dynavox Indi™ after a full charge can become shorter over time than when the device was new.

The Tobii Dynavox Indi™ device uses a Li-ion Polymer battery.

If you are in a hot environment, be aware that it can affect the ability to charge the battery. The internal temperature has to be between 0 °C/32 °F and 45 °C/113 °F for the battery to charge. If the internal battery temperature raises above 45 °C/113 °F the battery will not charge at all.

Move the Tobii Dynavox Indi™ device to a cooler environment to let the battery charge properly.

Avoid exposing the Tobii Dynavox Indi™ device to fire or to temperatures above 60 °C/140 °F. These conditions may cause the battery to malfunction, generate heat, ignite or explode. Be aware that it is possible, in a worst case scenario, for temperatures to reach greater than those stated above in, for example, the trunk of a car on a hot day. So, storing the Tobii Dynavox Indi™ device, in a hot car trunk could conceivably lead to a malfunction.

Only charge the Tobii Dynavox Indi™ battery in an ambient temperature of 0 °C/32 °F to 45 °C/113 °F.

Use only the supplied power adapter to charge the Tobii Dynavox Indi™ device. Using unauthorized power adapters may severely damage the Tobii Dynavox Indi™ device.

For safe operation of the Tobii Dynavox Indi™ device, use only charger and accessories approved by Tobii Dynavox.

Do not open, or modify, the casing of the Tobii Dynavox Indi™ device or of the power supply, since you may be exposed to potentially hazardous electrical voltage. The device contains no serviceable parts. If the Tobii Dynavox Indi™ device or it's accessories are mechanically damaged, **do not to use them**.

If the battery are not charged or the Tobii Dynavox Indi™ is not connected to the power supply, the Tobii Dynavox Indi™ device will shut down.

If the Power Supply Cord is damaged it needs to be replaced by Service Personnel only. Do not use the Power Supply Cord until replaced.

Disconnect the AC power plug of the Power adapter from the wall socket when not charging the device and disconnect the power cable from the device.

Special regulations apply to shipping devices containing Li-ion Polymer batteries. If dropped, crushed, or short-circuited, these batteries can release dangerous amounts of heat and may ignite, and are dangerous in fires.

Please reference IATA regulations when shipping lithium metal or lithium ion batteries or cells: http://www.iata.org/whatwedo/cargo/dangerous_goods/Pages/lithium_batteries.aspx

The Power adapters shall not be used without an adult or Caregiver supervision.

1.3 Temperature

Due to use in direct sunshine or in any other hot environment the Tobii Dynavox Indi™ device may have hot surfaces. There is a built in temperature sensor that monitors the temperature. If the sensor detects to hot internal temperature the device will automatically trigger a Windows Shut Down or Hibernate (depending on Windows Power Button configuration). It may take a while before Tobii Dynavox Indi™ device can be restarted as the unit may first need time to cool down.

1.4 Mounting

The Tobii Dynavox Indi™ should be mounted according to MANUFACTURER instructions of approved mounts. Tobii Dynavox or its agents are not liable for damage or injuries to a person or its property due to a Tobii Dynavox Indi™ falling from a mounted configuration. The mounting of a Tobii Dynavox Indi™ is done entirely at the user's own risk.

1.5 Emergency

Do not rely on the Tobii Dynavox Indi™ device for emergency calls or banking transactions. We recommend having multiple ways to communicate in emergency situations. Banking transactions should only be carried out with a system recommended by, and approved according to the standards of your bank.

1.6 Electricity

Do not open the casing of the Tobii Dynavox Indi™ device, since you may be exposed to potentially hazardous electrical voltage. The device contains no user serviceable parts.

1.7 Child Safety

The Tobii Dynavox Indi™ are advanced computer systems and electronic devices. As such they are composed of numerous separate, assembled parts. In the hands of a child certain of these parts have the possibility of being separated from the device, possibly constituting a choking hazard or another danger to the child.

Young children should not have access to, or the use of, the device without parental or guardian supervision.

1.8 Software

Using an antivirus program is strongly recommended.

Software other than that which is pre-installed on the Tobii Dynavox Indi™ is installed at the user's own risk. External software could cause the Tobii Dynavox Indi™ to malfunction and might not be covered by the warranty.

1.9 Magnetic Field

The Tobii Dynavox Indi™ device contains magnets. Magnetic fields may interfere with pacemakers, defibrillators, or other medical devices. As a general rule, maintain a minimum distance of 6 inches (15 centimeters) between any item with magnets and your heart device.

If you suspect that the Tobii Dynavox Indi™ device is interfering with your pacemaker or any other medical device, stop using the Tobii Dynavox Indi™ device and consult your physician for specific information about that affected medical device.

1.10 Third Party

Any use of the Tobii Dynavox Indi™ outside the intended use and together with any third party software or hardware that changes the intended use is a risk and nothing that Tobii Dynavox can take any responsibility for.

1.11 Environmental Control

The Programmable IR Remote Control — Environmental Control Unit (ECU) is not to be relied on as the only method of interaction with IR controlled devices.

2 Compliance Information

2.1 FCC Statement

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Modifications not expressly approved by Tobii Dynavox could void the user's authority to operate the equipment under FCC rules.

2.2 CE Statement

The Tobii Dynavox Indi™ Device is in compliance with the essential requirements and other relevant provisions of the following Directives. All essential radio test suites have been carried out:

- 2011/65/EU (RoHS 2) - Restriction of Hazardous Substances Directive
- R&TTE 1999/5/EC (R&TTE) - European Radio equipment and Telecommunications Terminal Equipment Directive
- 2014/30/EU (EMC) - Electromagnetic Compatibility Directive
- 2014/35/EU (LVD) - Low Voltage Directive
- 2009/125/EC (Eco Design) - Ecodesign Directive
- 2012/27/EC (Energy Efficiency) - Energy Efficiency Directive
- WEEE - Waste Electrical & Electronic Equipment
- REACH - Registration, Evaluation, Authorisation and restriction of Chemicals

CE0700

CE Compliance Authorization mark:

2.3 Industry Canada Statement

This Class B digital apparatus complies with Canadian ICES-003.

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2.4 IECEE CB Scheme

For Tobii Dynavox Indi™, CB Scheme is pending for additional markets (Australia, New Zealand, Argentina, Mexico, South Africa, China and others).

2.5 Radio Wave Exposure and Specific Absorption Rate (SAR) Information

The Tobii Dynavox Indi™ device was tested for typical body-worn operations. To comply with RF exposure requirements, a minimum separation distance of 0 mm must be maintained between the user's body and the handset, including the antenna. Third-party belt-clips, holsters, and similar accessories used by the Tobii Dynavox Indi™ device should not contain any metallic components. Body-worn accessories that do not meet these requirements may not comply with RF exposure requirements and should be avoided. Use only the supplied antenna.

The Tobii Dynavox Indi™ device complies with RF specifications when used at 0 mm from the body.

3 Customer Support

For support, please contact your local representative or Support at Tobii Dynavox. In order to receive assistance as quickly as possible, make sure you have access to your Tobii Dynavox Indi™ device and, if possible, an Internet connection. You should also be able to supply the serial number of the device, which you will find on the Service tag label.

For further product information and other support resources, please visit www.tobiidynavox.com or www.myTobiiDynavox.com.

4 Disposing of the Tobii Dynavox Indi™ Device

Do not dispose of the Tobii Dynavox Indi™ Device in general household or office waste. Follow your local regulations for the disposal of electrical and electronic equipment.

5 Technical Specifications

5.1 Tobii Dynavox Indi™ Device

Standard	Tobii Dynavox Indi™
Type	Speech Tablet PC
Model	Indi™
Manufacturer	Tobii Dynavox LLC
Screen	10.1", Aspect ratio: 16:10, LED Backlight Unit

Standard	Tobii Dynavox Indi™
Screen Resolution	1920 × 1200
Touch Panel	Multi point Capacitive Touch
Dimensions (WxHxD)	244 × 176 × 35 mm 9.6 × 6.9 × 1.4 inches
Weight	565 g 1.1 lbs
Speakers	2 × 31 mm 4 ohm, 4 W
Microphone	Analog Microphone
Processor	Intel Atom Z8350 – 4 cores, 2 MB cache, 1.92 Ghz
RAM	4 GB
Operating system	Microsoft® Windows 10 Pro 64-Bit
Flash drive	64 GB eMMC
Sensors	3-axis gyro, accelerometer, compass, ambient light sensor
Connectors	1 × DC in 12 VDC 2 A 1 × 3.5 mm" Headset/Audio-out port (stereo) with jack detection (Headphone and microphone port) 2 × Micro-B USB 2.0 1 × Micro-B USB 3.0
Buttons	1 × Power On 1 × Volume Up 1 × Volume Down
WLAN/Bluetooth®	IEEE 802.11 a/b/g/n Bluetooth® 4.0 / BLE
IR Remote Control (Environmental Control Unit)	Learning remote with integrated library of pre-programmed devices
Camera	Rear-facing: 2 Mega pixels Front-facing: 5 Mega pixels
Battery Run Time	> 8 hours. normal usage
Battery Technology	Li-ion Polymer rechargeable battery
Battery Capacity	29.6 Wh
Battery Charge Time	< 4 hours
Power Supply	12 VDC 2A AC adapter

5.2 Power Adapter

Item	Specification
Input voltage	100 to 240 VAC
Input current (max)	0.5 A
Input frequency	50 to 60 Hz
Output current	2 A
Rated output voltage	12 VDC

6 Approved Accessories

For information about the latest approved Accessories for the Tobii Dynavox Indi™, please visit www.TobiiDynavox.com or contact your local Tobii Dynavox reseller.

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- 2011/65/EU (RoHS 2) – Restriction of Hazardous Substances Directive
- R&TTE 1999/5/EC (R&TTE) – European Radio Equipment and Telecommunications Terminal Equipment Directive
- 2014/30/EU (EMC) – Electromagnetic Compatibility Directive
- 2014/35/EU (LVD) – Low Voltage Directive
- 2009/125/EC (Eco Design) – Ecodesign Directive
- 2012/27/EC (Energy Efficiency) – Energy Efficiency Directive
- WEEE – Waste Electrical & Electronic Equipment
- REACH – Registration, Evaluation, Authorisation and restriction of Chemicals

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