

Goal	Demonstrate ability to independently select symbol or action buttons through most appropriate selection method. • () will participate in emerging page set activities through desired selection method.
Set Up	Page Set: Choose any Emerging Page Set from Simple Selection, Scenes, Play Together, or Games. • Success Tip: Fill homepage with emerging page sets that provide a communicator a wide array of fun and engaging activities.
Initiate Activity	You: Let's start with this activity today. It will be fun. • Start activity and model the activity by selecting several buttons.
Maintain Turns	AAC: Selects an item on the page set activity You: I love this game. It's so much fun. • Comment on the activity page set, and tell them what they are doing. (parallel talk) You: Oh, you are doing great. • Wait for user reaction, indirectly prompt them to continue interacting with the page set. You: Let's do more, this is fun. • Give them more time to make selections and explore the page set.
Wrap Up	 You: That was fun. Let's do more. You navigate to home page, and see if user selects another activity page set. Give them time to make selections, and if user doesn't select page set on their own, use pointer to assist user in selecting new page set. AAC: Selects new page set.

Success Tip:

- In order for the user to have as many page sets available for them to choose from as possible, the homepage can be edited to add additional page sets that are of interest of the user.
- Adding a new page set to the home page:
 - o Open the quick menu
 - CTRL + M
 - Right click
 - Touch and hold
 - o Select "edit home page"
 - Select "add page sets"
 - o Select an appropriate level for the user (Emerging, Symbol, or Text)
 - Find the page set you would like to add
 - Select "add to home page"