

Goal	Navigational Skills
	 Demonstrates early developing navigational skills to include "next page" or "go back" within activity.
Set Up	Page Set: My Book Section Time Content to make the head many inefaltoness.
	Success Tip: Customize Content to make the book meaningful to user.
Initiate Activity	You: Let's look at your book. You select the move arrow
Maintain Turns	 You: Let's start with "my family" Lift your hand slowly. Make sure they are watching you. Select My Family. They will learn by observing how you use their communication device. Read and comment about items on the page. You: Let's look at another page, "Go Back" Point to the back button while you are speaking, wait to see if they will select back button. If not, lift your hand slowly. Make sure they are watching and select the "back" button. They will learn by observing how you use their communication device. You: Let's read about "my likes" Select the button and read and comment about the page. You: Let's look at another page, "Go Back" Point to the back button while you are speaking, wait to see if they will select back button. If not, lift your hand slowly and model use. They will learn by observing. AAC: Selects "back" button.
Wrap Up	You: Great! What else can we read about you. AAC: Selects the next page.

Success Tip: Customize Content in your Book

- 1. Open the Quick Menu and Select Edit Button tool
- 2. Choose a button outlined in red to edit
- 3. Change the:
 - Image: Type in the button text and symbol search for the image you want in your book.
 - Sound: Let's make this easy and select "Synthesized Speech"
 - Link: Keep this tab "as is" ... Other. Next page.
- 4. Now select OK and move to the next picture and edit that next page until you are done with the book
- 5. When you are done you need to save your work
 - Open Quick menu
 - Choose "Save As"
 - Type in "My Book
 - Select "Save" and Select "Yes, in addition to the old version"
 - Select OK