

Instructions for Tobii batterybox.




- 1** Chargeoutlet.
- 2** Fuse 10A .
- 3** Connection outlet to unit.
- 4** On/Off button with green indication light.

Usage

- 1** Mount the battery box on the wheelchair.
- 2** Connect the cable from the unit to the connection outlet (3) on the battery box.
- 3** Press the green On/Off button (4), On will be indicated by a green light.
- 4** Start the computer.

Charging

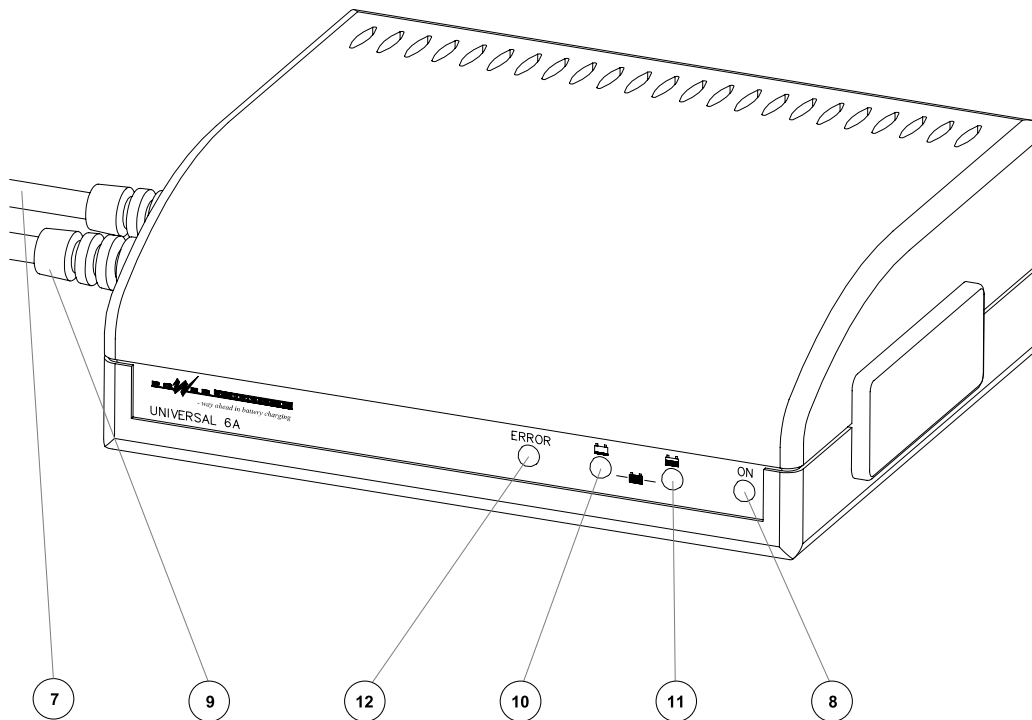

- 1** Turn off the P10 device.
- 2** Shut down the battery box by pressing the On/Off button (4) so that the green light goes out.
- 3** Disconnect the outlet cable (3) from the P10 before charging the battery
- 4** Connect the chargers power cable (7) to a wall socket. Indicator light (8) will turn on.
- 5** Connect the chargers battery cable (9) to the battery box charge cable (1). The indicator light (10) will turn on and the charge begins.
- 6** When the battery is charged to 80% the charge indicator light (11) turns on and the indicator light (10) turns off.
- 7** When the battery box is fully charged, both charge indicator lights (10-11) turn on.
- 8** Disconnect the charger from the battery box before connecting the P10 device back to the battery.