


MyTobii Installation


How to install and/or upgrade MyTobii software:

1. To download the latest MyTobii release, please follow this link:
http://www.tobii.com/assistive_technology/support_downloads/downloads.aspx
Extract the installation package by right clicking the downloaded zip file.
2. Close all applications including MyTobii Software.
3. Stop the TET Server by right clicking the TET Server tray icon (the little blue t) in the bottom right corner of your monitor and select "Stop".

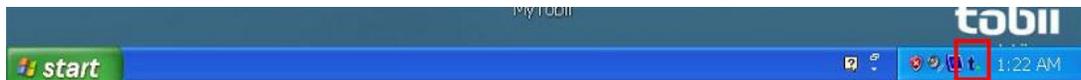


To right-click on the P10, press anywhere on the screen and hold for 1 second.


4. Uninstall the TET Server from Start, Control panel, Add/Remove programs. Choose "Tobii Eye Tracker Server" in the list of installed programs and then click on the Remove button.

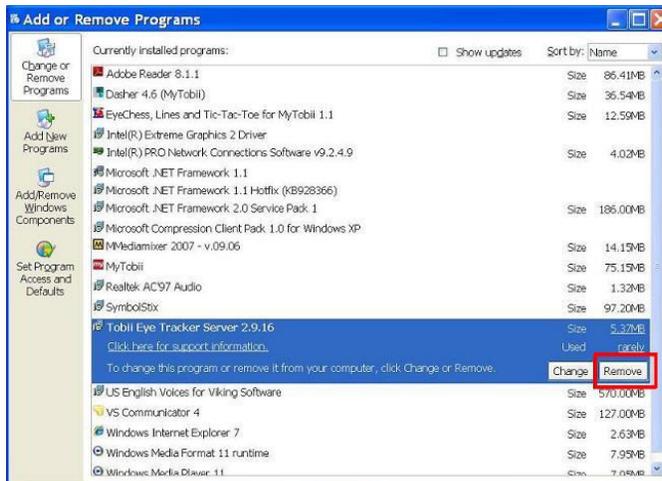


- Uninstall MyTobii from Start, Control panel, Add/Remove programs. Choose “MyTobii” in the list of installed programs and then click on the Remove button.

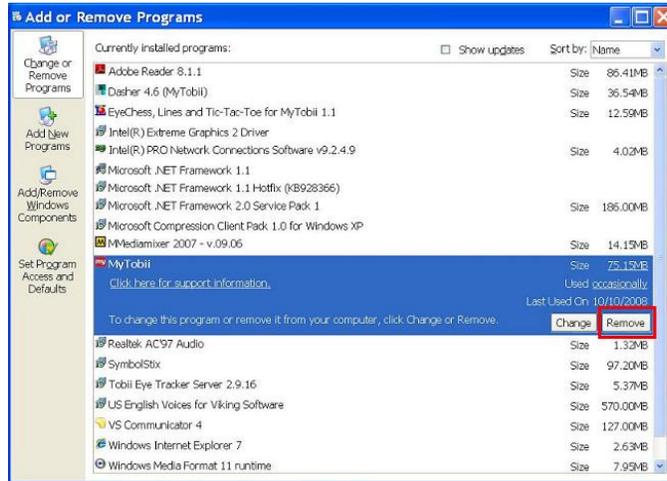

- Install the new version of MyTobii. Make sure to use the setup.exe file – it includes both the TET Server and MyTobii.

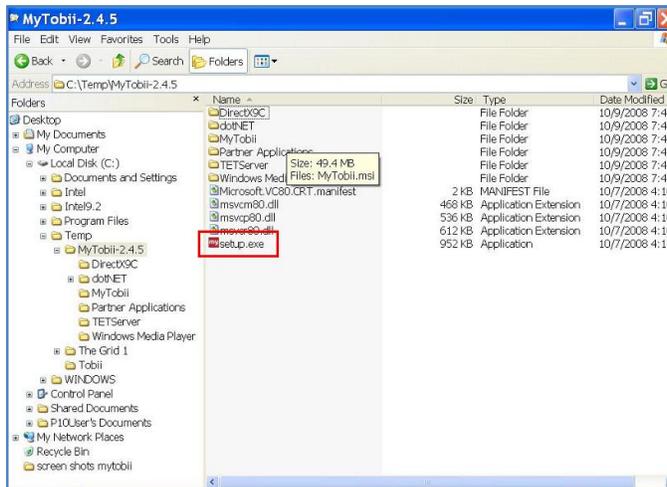

7. Verify the new versions in the Control panel – both TET Server and MyTobii. Choose “MyTobii” in the list of installed programs and then click on “Click here for support information” and the Support Info window will appear and there the version number is shown.

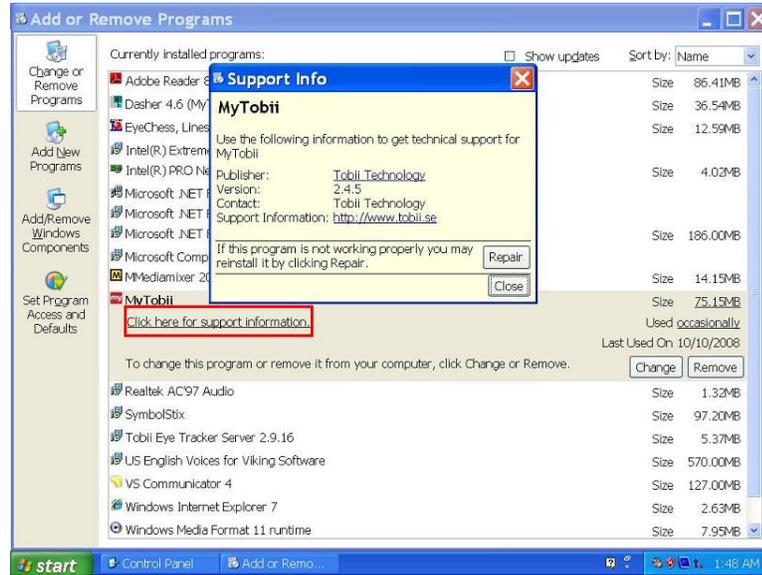

8. Restart the computer.

