

Release Notes

PRODUCT: MyTobii software
RELEASE NUMBER: MyTobii-2.4.5
RELEASE DATE: 2008/10/22
CUSTOMER SUPPORT: For more information or support, please visit www.tobii.com or email us at support@tobii.com

WHO SHOULD UPGRADE?

This release is mainly a bug fix release, but it also contains a new language, Turkish. Any user that experiences problems with the MyTobii e-mail client or with the feedback dot should upgrade to this version.

WHAT'S NEW IN THIS RELEASE

FEATURES

- New language: Turkish

BUG FIXES

- **Feedback dot**
 - The feedback dot is now always shown correctly on top of 3rd party applications. (Internal bug ID: MYT-229)
 - The feedback dot does not disappear when 3rd party applications do not terminate as they should (Internal bug ID: MYT-232)
- **MyTobii e-mail client**
 - A special keyboard for email addresses has been added. It is a modified English QWERTY keyboard which includes the @-sign and the underscore character. (Internal bug ID: MYT-165)
 - Loosing e-mails should no longer occur when using BOTH the MyTobii e-mail client and the VS Communicator 4 Premium (or VSC 3 Pro) e-mail client. (Internal bug ID: MYT-211)
 - The MyTobii software now launches fast even when having large amounts of emails. (Internal bug ID: MYT-236)
 - The MyTobii e-mail client now receives new messages in a more optimized way. (Internal bug ID: MYT-242)
 - The issue with duplicated e-mail messages in the inbox is fixed. (Internal bug ID: MYT-243)
 - When replying to an email, the body of the previous mail now has a ">" sign before all new lines. (Internal bug ID: MYT-244)
 - The MyTobii e-mail client now accepts attachments with the file extension ".rar". (Internal bug ID: MYT-247)



- **3rd part application management**
 - The MyTobii software now handles registering of 3rd party applications buttons in a more stable way. (Internal bug ID: MYT-237, MYT-238)
 - The MPA SDK now handles outgoing RCW calls in a more stable way. (Internal bug ID: MYT-240)
 - The MyTobii software now handles partner applications that are closed properly in a more stable way. (Internal bug ID: MYT-249)
- **Document Browser**
 - The characters å, ä, ö, æ, ø are now displayed in a correct way (apart from documents saved in ANSI encoding). (Internal bug ID: MYT-225)
- **Other bug fixes**
 - Off screen menus do not flash by when starting MyTobii anymore. (Internal bug ID: MYT-228)
 - The MyTobii software now has a more stable gaze event handling. (Internal bug ID: MYT-239)
 - The onscreen keyboard window is now set to the foreground window for each key press event. This is to prevent key events to be sent to the wrong window when using a secondary monitor display. (Internal bug ID: MYT-245)

DOWNLOAD

To download this MyTobii release, please follow this link:

http://www.tobii.com/assistive_technology/support_downloads/downloads.aspx

DISCUSSION FORUM

If you are still not a member of the MyTobii Community, then you should register right away and benefit from other users experiences as well! We look forward to receiving your feedback about both the MyTobii hardware and software.

<http://www.mytobiicommunity.com/>

REPORTING BUGS

To report a bug in the MyTobii software, please follow this link:

http://www.tobii.com/assistive_technology/support_downloads/support.aspx

