## Safety and Compliance PCEye Plus by Tobii Dynavox

### 1 Safety

#### 1.1 Mounting Warning

The PCEye Plus should be mounted according to the instructions of the approved mounts used. Tobii AB or its agents are not liable for damage or injuries to a person or its property due to a PCEye Plus falling from a mounted configuration. The mounting of a PCEye Plus is done entirely at the user's own risk.

The double sided tape supplied with the adhesive Magnetic Mounting Plates (if applicable) is designed to permanently attach the bracket to the monitor/laptop. Attempting to remove the bracket after attaching it using the supplied tape can result in damage both to the monitor/laptop and the bracket.

Do not lick the adhesive tape or place any part of the tape or Magnetic Mounting Plate in one's mouth or on one's body.

Do not mount the PCEye Plus on monitors positioned above the head or face of a user unless using a dedicated "fixed mounting" solution.

#### 1.2 Epilepsy Warning

Some people with **Photosensitive Epilepsy** are susceptible to epileptic seizures or loss of consciousness when exposed to certain flashing lights or light patterns in everyday life. This may happen even if the person has no medical history of epilepsy or has never had any epileptic seizures.

A person with Photosensitive Epilepsy would also be likely to have problems with TV screens, some arcade games, and flickering fluorescent bulbs. Such people may have a seizure while watching certain images or patterns on a monitor, or even when exposed to the light sources of an eye tracker. It is estimated that about 3-5% of people with epilepsy have this type of Photosensitive Epilepsy. Many people with Photosensitive Epilepsy experience an "aura" or feel odd sensations before the seizure occurs. If you feel odd during use, move your eyes away from the eye tracker.

#### 1.3 Infrared Warning

When activated, the PCEye Plus emits pulsed infrared (IR) light. Certain medical devices are susceptible to disturbance by IR light and/or radiation. Do not use the PCEye Plus when in the vicinity of such susceptible medical devices as their accuracy or proper functionality could be inhibited.

#### 1.4 Magnetic Field Warning

The PCEye Plus Magnetic Mounting contains magnets. Magnetic fields may interfere with the function of cardiac pacemakers and implantable cardioverter-defibrillators. As a general rule, maintain a minimum distance of 6 inches (15 centimeters) between any item with magnets and your heart device.

#### 1.5 Child Safety

The PCEye Plus is an advanced computer system and electronic device. As such it is composed of numerous separate, assembled parts. In the hands of a child certain of these parts have the possibility of being separated from the device, possibly constituting a choking hazard or another danger to the child.

Young children should not have access to, or use, the device without parental or guardian supervision.

#### 1.6 Do not open PCEye Plus

Non-compliance will result in loss of Warranty! There are no user serviceable components inside. Contact Tobii Dynavox Support if your PCEye Plus is not working properly.

#### 1.7 Emergency

Do not rely on the device for emergency calls or banking transactions. We recommend having multiple ways to communicate in emergency situations. Banking transactions should only be carried out with a system recommended by, and approved according to the standards of, your bank.

#### 1.8 Windows Control

Some people may experience a certain amount of fatigue (due to intentional eye focusing and hard concentration) or even a dryness of the eyes (due to less frequent blinking) when first getting used to Windows Control. If you are experiencing fatigue or dry eyes start off slowly and limit the length of your Windows Control sessions to your comfort level. Remoisturizing eye drops can be helpful to combat dryness.

#### 1.9 Third Party

Any use of the PCEye Plus outside the intended use and together with any third party software or hardware that changes the intended use is a risk and nothing that Tobii AB can take any responsibility for.

### 2 Compliance Information

The PCEye Plus is CE-marked, indicating compliance with the essential health and safety requirements set out in European Directives..

#### 2.1 FCC Statement

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Modifications not expressly approved by Tobii AB could void the user's authority to operate the equipment under FCC rules.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications.

However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into a wall outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or experienced radio/TV technician for assistance.

#### 2.2 Industry Canada Statement

This Class B digital apparatus complies with Canadian ICES-003.

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#### 2.3 CE Statement

The PCEye Plus complies with the following Directives:

- 2014/30/EU (EMC) - Electromagnetic Compatibility Directive
- 2011/65/EU (RoHS) - Restriction of Hazardous Substances Directive

#### 2.4 Standards

The PCEye Plus complies with the following standards:

- FCC part 15, Class B
- ICES-003 Issue 6:2016 Class B
- EN 55032: 2012+AC: 2013 Class B
- EN 55024:2010+A1:2015
- EN 61000-3-2: 2014

- EN 61000-3-3: 2013
- AS/NZS CISPR 32:2013

### 3 Customer Support

For support, please contact your local representative or Tobii Dynavox Support. In order to receive assistance as quickly as possible, make sure you have access to your PCEye Plus and, if possible, an Internet connection. You should also be able to supply the serial number of the device, which you find on the back of the device. Find our Support pages online at: [www.tobiidynavox.com](http://www.tobiidynavox.com) or [www.myTobiiDynavox.com](http://www.myTobiiDynavox.com).

### 4 Disposing of the PCEye Plus

Do not dispose of the PCEye Plus in general household or office waste. Follow your local regulations for the disposal of electrical and electronic equipment.

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