

## Release Notes **Tobii Dynavox PCEye Software**

### Introduction

The Tobii Dynavox PCEye Software is a software solution for setting up and configuring use of Gaze Interaction with the Tobii Dynavox PCEye Go, PCEye Explore and PCEye Mini eye trackers.

### System Requirements

- Windows Vista (SP2) 7, 8, 8.1 or 10
- .NET Framework 4 Full
- DirectX 9.0c

### Supported Tobii Dynavox Eye Trackers

| Discontinued Support | Final Version |
|----------------------|---------------|
| PCEye                | 1.3.0         |

  

| Supported Products | From version |
|--------------------|--------------|
| PCEye Go           | 2.0.0        |
| EyeMobile          | 2.1.0        |
| PCEye Mini         | 2.4.0        |
| PCEye Explore      | 2.4.0        |
| EyeMobile Mini     | 2.4.0        |

**Customer support:** For more information or support, please visit [www.TobiiDynavox.com](http://www.TobiiDynavox.com).

### Release Version 2.4.2

**Release date:** 2016-11-03

#### Improvements

- More robust detection of tablet orientation for PCEye Configuration Guide (CX-715)
- PCEye Notifier's Exit context menu option is removed (CX-721)
- PCEye Notifier's startup made more robust when starting several instances simultaneously (CX-735)
- Version 2.8.3 of Gaze Interaction Software included

### Release Version 2.4.1

**Release date:** 2016-08-15

#### Improvements

- Fixed a bug in the installer that prevented .NET 4.5 from being installed if .NET 4 was already installed (CX-707)

### Support for Your Tobii Dynavox Device

#### Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: [www.TobiiDynavox.com](http://www.TobiiDynavox.com) or [www.myTobiiDynavox.com](http://www.myTobiiDynavox.com).

#### Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit [www.TobiiDynavox.com/contact](http://www.TobiiDynavox.com/contact)

- Improved PCEye Notifier behavior in conjunction to TGIS upgrade (CX-708)
- Upgraded Tobii Service version to fix a bug related to using the camera on Microsoft Surface tablets (CX-712)

## Release Version 2.4.0

**Release date:** 2016-05-12

### New Features

- Support for PCEye Explore Eye Tracker (CX-664)
- Support for PCEye Mini Eye Tracker (CX-665)
- Support for EyeMobile Mini (CX-666)
- Version 2.8.1 of Gaze Interaction Software included

### Improvements

- User Manuals are moved to the web to decrease installer size (CX-686)

## Release Version 2.3.2

**Release date:** 2016-01-25

### Improvements

- Version 2.7.4 of Gaze Interaction Software included

## Release Version 2.3.1

**Release date:** 2016-01-11

### New Features

- Version 2.7.3 of Gaze Interaction Software included
- Version 1.4.2 of Update Notifier included

### Improvements

- Product information is made persistent on the target machine (CX-644)
- Support link PCEye Configuration Guide in Add or Remove Programs now shows [www.tobiidynavox.com](http://www.tobiidynavox.com) (CX-646)
- Added predefined measurements for Microsoft Surface Pro 4 in PCEye Configuration Guide (CX-625)

## Release Version 2.3.0

**Release date:** 2015-09-15

### New Features

- Support for Windows 10
- Version 2.7.0 of Gaze Interaction Software included

## Support for Your Tobii Dynavox Device

### Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: [www.TobiiDynavox.com](http://www.TobiiDynavox.com) or [www.myTobiiDynavox.com](http://www.myTobiiDynavox.com).

### Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit [www.TobiiDynavox.com/contact](http://www.TobiiDynavox.com/contact)

- Version 1.4.0 of Update Notifier included
- Installer for Gaze Selection word prediction language packs included

## Improvements

- Name change from Tobii to Tobii Dynavox. (CX-623)
- Improved error handling when the PCEye Configuration Guide is running without Gaze Interaction Software installed. (CX-639)

## Release Version 2.2.4

**Release date:** 2014-09-20

## New Features

- Version 2.6.1 of Tobii Gaze Interaction Software included
- Version 1.3.1 of PCEye Update Notifier included
- Updated the user manuals with illustrations of new magnetic mount (CX-613)

## Improvements

- Resolved issue where a non-executable file was set to run at Windows startup (CX- 610)

## Release Version 2.2.3

**Release date:** 2014-05-16

## Improvements

- Updated the user manuals. (CX-601)
- Added device data for Microsoft Surface 2. (CX-590)
- Added device data for Dell Venue 11 Pro 5130. (CX-590)
- Added device data for Dell Venue 11i Pro 7130/7139. (CX-590, CX-611)

## Release Version 2.2.2

**Release date:** 2013-12-02

## Improvements

- Updated the user manuals. (CX-592)
- Added device data for Microsoft Surface Pro 2. (CX-593)

## Release Version 2.2.1

**Release date:** 2013-11-06

## Support for Your Tobii Dynavox Device

### Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: [www.TobiiDynavox.com](http://www.TobiiDynavox.com) or [www.myTobiiDynavox.com](http://www.myTobiiDynavox.com).

### Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit [www.TobiiDynavox.com/contact](http://www.TobiiDynavox.com/contact)

## Improvements

- Updated the pictures in the Installation Guide to match the new mounting plate. (CX-586)
- Corrected the metadata of the self extracting installer. (CX-576)

## Release Version 2.2.0

**Release date:** 2013-10-11

## New Features

- New version of Tobii Gaze Interaction Software; 2.5.0.
- New version of EyeTrackerBrowser; 3.0.83.
- Update Notifier. This application notifies the user of available software updates and installs these updates.

## Improvements

- Improved values for the Positioning Guide track box. (CX-440)
- Fixed a bug where PCEye Configuration guide could launch Windows Control unexpectedly. (CX-538)

## Release Version 2.1.0

**Release date:** 2013-09-06

## New Features

- Installs Tobii Gaze Interaction Software 2.4.0.
- Support for EyeMobile.

## Improvements

- .NET 3.5 now included in PCEye Installation Guide. (CX-399)
- First page next button in PCEye Installation Guide is now disabled until the user selects a product. (CX-528)
- Improved performance of track status in PCEye Configuration Guide. (NESDK-1082)
- The Configuration Guide now only shows one confirmation regardless of how many time the save screen size button is pressed. (PCEYE-31)
- The Configuration Guides Screen and Calibration pages are now enabled/disabled properly with attached/removed EyeTracker. (PCEYE-47)
- The Installation- and Configuration Guides are now always possible to exit when they are complete. (PCEYE-48)
- Tobii Gaze Interaction Settings might crash if the process is started as another Windows user than the owner of the active session. (CX-439, CX-464)
- Server communication failure if Tobii Gaze Interaction Software is uninstalled prior to PCEye software while Notifier.exe is running. (CX-509)
- The PCEye Installation Guide window is too large when using 125% scaling on devices with low screen resolution. (CX-531)

## Release Version 2.0.1

**Release date:** 2013-07-08

## Support for Your Tobii Dynavox Device

### Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: [www.TobiiDynavox.com](http://www.TobiiDynavox.com) or [www.myTobiiDynavox.com](http://www.myTobiiDynavox.com).

### Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit [www.TobiiDynavox.com/contact](http://www.TobiiDynavox.com/contact)

## New Features

- Installs Tobii Gaze Interaction Software 2.3.1.

## Improvements

- Support for the old PCEye in the new PCEye Configuration Guide. (CX-435)
- The USB installation stick now only shows one installation option for the user. (CX 510, PCEYE-8)
- Manuals are accessible on the USB installation stick prior to installation. (PCEYE-9)
- Updated the text "remove USB now please" to "Please remove the Tobii PCEye USB flash drive.". (CX-511)

## Release Version 1.3.0

**Release date:** 2012-08-29

## Improvements

- The Installation Guide, Configuration Guide and Tobii Notifier are all upgraded to use .NET 4 (Internal id: CX-329)
- Upgrade to use Tobii Eye Control Suite 2 (Internal id: CX-328)
- The default Windows Control method for PCEye is now Tobii Gaze Selection (Internal id: CX-330)
- The texts in the PCEye Installation and Configuration Guides become invisible when using "High Contrast Black" color scheme (Internal id: CX-314)
- The input fields for centimeters and inches on the screen measurement page in the PCEye Configuration Guide interact with each other (Internal id: CX-332)
- It is possible to make the PCEye Configuration Guide accept the screen size 0 x 0 cm (Internal id: CX-334)
- Tobii Notifier does not start up automatically after PCEye Configuration Guide (Internal id: CX-338)

## Support for Your Tobii Dynavox Device

### Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: [www.TobiiDynavox.com](http://www.TobiiDynavox.com) or [www.myTobiiDynavox.com](http://www.myTobiiDynavox.com).

### Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit [www.TobiiDynavox.com/contact](http://www.TobiiDynavox.com/contact)