

Release Notes **Tobii Dynavox Snap + Core First**

Release Version 1.3

Release date: 2018-03-08

New Features

- Full German language support, including Core First Page Set, search, morphology, and voices.
 - German METACOM symbols are included with German app and device purchases, and are otherwise available through in-app purchase.
- The following Acapela German voices are now available in Snap: Lea, Claudia, Claudia Smile, Julia, Sarah, Jonas, Andreas, and Klaus.
- A new subscription option (monthly or annual) for speech functionality (Windows and iOS) is available through the myTobiiDynavox.com store.
- You can now purchase speech functionality for the Windows app through a printed coupon. This enables sales through purchase orders and resellers and can be redeemed through the myTobiiDynavox store.

Improvements

- Bug fixes.

Release Version 1.2

Release date: 2017-12-12

New Features

- Introduction of Tobii Dynavox's first truly bilingual page set/user — Core First English/Spanish Bilingual.
 - All pages of the Core First English/Spanish Bilingual page set are available in Spanish and English.
 - Update all pages from English to Spanish, and from Spanish to English with a single button selection.
- Customers may select language-specific voices to support on the fly voice switching:
 - When the Message Window contains both English and Spanish text, the English text is spoken by the defined English voice and the Spanish text is spoken by the defined Spanish voice.
 - On the fly voice switching may also be used for speech from buttons.
- The following Acapela Spanish voices are now available in Snap: Antonio (Spain), Emilio (Spanish US), Ines (Spain), Maria (Spain), Rodrigo (Spanish US), Rosa (Spanish US), Valeria (Spanish US).
 - Two of the Spanish voices have matching English voices: Emilio English and Valeria English.
- The language of the Settings/Editing menu follows the language of the operating system, i.e., the Snap interface will be in Spanish if the OS is Spanish (US), Spanish (MX), Spanish (LA), or Spanish (ES).

Improvements

- The Setup Wizard has been updated for the bi-lingual user and ease of use.
- Automatic agreement added for Message Window text, for example, when the word 'a' is in the Message Window, then adding the word 'apple' updates 'a' to 'an.'

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com/support-training

Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact

- The myTobiiDynavox password is now cleared when switching users.
- Bug fixes.

Release Version 1.1

Release date: 2017-10-09

New Features

- Import, export, share, and Sync your content.
- Integration with myTobiiDynavox for import, export, share, and Sync.
- Support for Windows Control 2, EyeMobile Plus, and PCEye Plus.
- New Message Window options in Page Set > Style:
 - font size
 - text color
- Added “Delay between selections” setting for Touch Exit and Touch Enter.
- Added the action “Launch Communicator.”
- Updates to Visit functionality, including three new navigation actions:
 - Cancel Visit
 - Extend Visit
 - Return from Visit
- Updated functionality for the free version of the app:
 - Companion app for Tobii Dynavox device users
 - SLPs can obtain the full functionality of the app with their valid ASHA code
 - Those not covered by the instances above may obtain a 30-day free trial of the speaking app (via myTobiiDynavox)
 - In-app purchase of speech
- Choice of content language/culture through the Setup Wizard.
 - New content for the U.K.

Improvements

- Updated Setup Wizard interface.
- Performance improvements.
- Bug fixes.

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com/support-training

Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact