

Release Notes **Tobii Dynavox Snap + Core First**

Release Version 1.4.3.4231

Note: On Windows this appears as version **1.4.4231**

Release date: 2018-07-25

Improvements

- Bug fixes.

Release Version 1.4.2

Release date: 2018-07-11

New Features

- Snap now has full support for Spanish (Latin America).
 - Latin American Spanish Page Sets: Core First and blank.

Improvements

- New User Wizard additions.
- UI string updates to Swedish and Spanish.
- Bug fixes.

Release Version 1.4

Release date: 2018-06-18

New Features

- Snap now has full support for Swedish.
 - The following Acapela Swedish voices are now available: Mia, Kal, Samuel (Swedish Finland), Freja, Elin, Emma, Filip, Emil, and Erik.
- Snap now has lite support for Dutch.
 - The following Acapela Dutch voices are now available: Femke, Jasmijn, Daan, and Max.
- The new button type, **Chat**, can be used to toggle Chat Mode on and off. When Chat Mode is toggled on, the current content of the Message Window is cleared and saved. While Chat Mode is on, the Message Window may be used for conversation. When Chat Mode is toggled off, the saved Message Window content is restored.
- The new Page Set Style setting, **Grid Margin Factor**, controls the amount of space between buttons.
- Added the **Edit Mode Language** System setting, which changes the language of the Editing Panel.
- The new User Preference **Clear after Speech** automatically clears the Message Window when text is inserted after the Message Window has been spoken.
- **Delay after Page Change** is a new feature for Gaze Interaction that allows the user to view newly opened pages without making eye gaze selections for up to 10 seconds.

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com/support-training

Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact

- The new button type **Pause Access Method** can be used to pause and un-pause the Access Methods **Mouse Dwell** and **Gaze Interaction**.
- The new Page Set Style setting **Link Visualization** includes options to display all “link buttons” with Navigation Icons (the default), in a folder shape, or with no distinguishing style.
- The **Passcode** feature now has options to password protect specific features (Edit Mode, Sync, Vocabulary Search).

Improvements

- Bug fixes.

Release Version 1.3

Release date: 2018-03-08

New Features

- Full German language support, including Core First Page Set, search, morphology, and voices.
 - German METACOM symbols are included with German app and device purchases, and are otherwise available through in-app purchase.
- The following Acapela German voices are now available in Snap: Lea, Claudia, Claudia Smile, Julia, Sarah, Jonas, Andreas, and Klaus.
- A new subscription option (monthly or annual) for speech functionality (Windows and iOS) is available through the myTobiiDynavox.com store.
- You can now purchase speech functionality for the Windows app through a printed coupon. This enables sales through purchase orders and resellers and can be redeemed through the myTobiiDynavox store.

Improvements

- Bug fixes.

Release Version 1.2

Release date: 2017-12-12

New Features

- Introduction of Tobii Dynavox’s first truly bilingual page set/user — Core First English/Spanish Bilingual.
 - All pages of the Core First English/Spanish Bilingual page set are available in Spanish and English.
 - Update all pages from English to Spanish, and from Spanish to English with a single button selection.
- Customers may select language-specific voices to support on the fly voice switching:
 - When the Message Window contains both English and Spanish text, the English text is spoken by the defined English voice and the Spanish text is spoken by the defined Spanish voice.
 - On the fly voice switching may also be used for speech from buttons.
- The following Acapela Spanish voices are now available in Snap: Antonio (Spain), Emilio (Spanish US), Ines (Spain), Maria (Spain), Rodrigo (Spanish US), Rosa (Spanish US), Valeria (Spanish US).
 - Two of the Spanish voices have matching English voices: Emilio English and Valeria English.

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com/support-training

Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact

- The language of the Settings/Editing menu follows the language of the operating system, i.e., the Snap interface will be in Spanish if the OS is Spanish (US), Spanish (MX), Spanish (LA), or Spanish (ES).

Improvements

- The Setup Wizard has been updated for the bi-lingual user and ease of use.
- Automatic agreement added for Message Window text, for example, when the word 'a' is in the Message Window, then adding the word 'apple' updates 'a' to 'an.'
- The myTobiiDynavox password is now cleared when switching users.
- Bug fixes.

Release Version 1.1

Release date: 2017-10-09

New Features

- Import, export, share, and Sync your content.
- Integration with myTobiiDynavox for import, export, share, and Sync.
- Support for Windows Control 2, EyeMobile Plus, and PCEye Plus.
- New Message Window options in Page Set > Style:
 - font size
 - text color
- Added "Delay between selections" setting for Touch Exit and Touch Enter.
- Added the action "Launch Communicator."
- Updates to Visit functionality, including three new navigation actions:
 - Cancel Visit
 - Extend Visit
 - Return from Visit
- Updated functionality for the free version of the app:
 - Companion app for Tobii Dynavox device users
 - SLPs can obtain the full functionality of the app with their valid ASHA code
 - Those not covered by the instances above may obtain a 30-day free trial of the speaking app (via myTobiiDynavox)
 - In-app purchase of speech
- Choice of content language/culture through the Setup Wizard.
 - New content for the U.K.

Improvements

- Updated Setup Wizard interface.
- Performance improvements.
- Bug fixes.

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com/support-training

Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact