Safety and Compliance, Speech Case

1 Safety

The Speech Case device has been tested and approved as compliant to all the Specifications and Standards listed in 2 *Compliance Information, page 2* of this manual and in the 6 *Technical Specifications, page 3*. Nevertheless, in order to ensure safe operation of your Speech Case, there are a few safety warnings to bear in mind:

No modification of this equipment is allowed.

The user may never attempt to change the battery. Altering of the battery may cause risk of explosion.

The Speech Case is not to be used as a life supporting device, and it shall not be relied on in case of loss of function due to power loss or other causes.

There could be a choking hazard risk if small parts detach from the Speech Case device.

The strap and charging cable could present strangulation hazards to young children. Never leave small children unattended with the strap or charging cable.

The Speech Case device shall not be exposed to or used in rain or weather conditions outside the Technical Specification of the Speech Case device.

Young children or people with cognitive disabilities should not have access to, or use, the Speech Case device without parental or guardian supervision.

The Speech Case device shall be used with care when moving around.

1.1 Avoiding Hearing Damage

Permanent hearing loss may occur if earphones, headphones or speakers are used at high volume. To prevent this, the volume should be set to a safe level. You can become desensitized over time to high sound levels which may then sound acceptable yet still could be damaging your hearing. If you experience symptoms such as ringing in your ears, please lower the volume or stop using the earphones/headphones. The louder the volume, the less time is required before your hearing could be affected.

Hearing experts suggest the following measures to protect your hearing:

- Limit the amount of time you use earphones or headphones at high volume.
- Avoid turning up the volume to block out noisy surroundings.
- Turn the volume down if you cannot hear people speaking near you.

To establish a safe volume level:

- Set your volume control at a low setting.
- Slowly increase the sound until you can hear it comfortably and clearly, without distortion.

1.2 Power Supply and Batteries

The power source shall conform to the requirement of the Safety Extra Low Voltage (SELV) standard, and supply power with rated voltage which conforms to Limited power Source requirement according to IEC60950-1.

The Speech Case device contains a rechargeable battery. All rechargeable batteries degrade over time. Thus the possible usage times for the Speech Case after a full charge can become shorter over time than when the device was new.

The Speech Case device uses a Li-ion Polymer battery.

If you are in a hot environment, be aware that it can affect the ability to charge the battery. The internal temperature has to be between 0 °C/32 °F and 45 °C/113 °F for the battery to charge. If the internal battery temperature raises above 45 °C/113 °F, the battery will not charge at all.

Move the Speech Case device to a cooler environment to let the battery charge properly.

Avoid exposing the Speech Case device to fire or to temperatures above 60 °C/140 °F. These conditions may cause the battery to malfunction, generate heat, ignite or explode. Be aware that it is possible, in a worst case scenario, for temperatures to reach greater than those stated above in, for example, the trunk of a car on a hot day. So, storing the Speech Case device, in a hot car trunk could conceivably lead to a malfunction.

Only charge the Speech Case battery in an ambient temperature of 0 °C/32 °F to 45 °C/113 °F.

Use only the supplied power adapter to charge the Speech Case device. Using unauthorized power adapters may severely damage the Speech Case device.

For safe operation of the Speech Case device, use only the charger and accessories approved by Tobii Dynavox.

Do not open, or modify, the casing of the Speech Case device or of the power supply, since you may be exposed to potentially hazardous electrical voltage. The device contains no serviceable parts. If the Speech Case device or its accessories are mechanically damaged, **do not use them**.

If the battery is not charged or the Speech Case is not connected to the power supply, the Speech Case device will shut down.

If the Power Supply Cord is damaged, it needs to be replaced by Service Personnel only. Do not use the Power Supply Cord until replaced.

Disconnect the AC power plug of the Power adapter from the wall socket when not charging the device and disconnect the power cable from the device.

Special regulations apply to shipping devices containing Li-ion Polymer batteries. If dropped, crushed, or short-circuited, these batteries can release dangerous amounts of heat, may ignite, and may be dangerous in fires.

Please reference IATA regulations when shipping lithium metal or lithium ion batteries or cells: http://www.iata.org/whatwedo/cargo/dangerous_goods/Pages/lithium_batteries.aspx

The Power adapter shall not be used without an adult or caregiver supervision.

1.3 Temperature

Due to use in direct sunshine or in any other hot environment, the Speech Case device may have hot surfaces.

1.4 Mounting

The Speech Case should be mounted according to MANUFACTURER instructions of approved mounts. Tobii Dynavox or its agents are not liable for damage or injuries to a person or its property due to a Speech Case falling from a mounted configuration. The mounting of a Speech Case is done entirely at the user's own risk.

1.5 Emergency

Do not rely on the Speech Case device for emergency calls. We recommend having multiple ways to communicate in emergency situations.

1.6 Electricity

Do not open the casing of the Speech Case device, since you may be exposed to potentially hazardous electrical voltage. The device contains no user serviceable parts.

1.7 Child Safety

The Speech Case is an advanced computer systems and electronic device. As such they are composed of numerous separate, assembled parts. In the hands of a child certain of these parts have the possibility of being separated from the device, possibly constituting a choking hazard or another danger to the child.

Young children should not have access to, or the use of, the device without parental or guardian supervision.

1.8 Magnetic Field

The Speech Case device contains magnets. Magnetic fields may interfere with pacemakers, defibrillators, or other medical devices. As a general rule, maintain a minimum distance of 6 inches (15 centimeters) between any item with magnets and your heart device.

If you suspect that the Speech Case device is interfering with your pacemaker or any other medical device, stop using the Speech Case device and consult your physician for specific information about that affected medical device.

1.9 Third Party

Any use of the Speech Case outside the intended use and together with any third party software or hardware that changes the intended use is a risk and nothing that Tobii Dynavox can take any responsibility for.

2 Compliance Information

The Speech Case is CE-marked, indicating compliance with the essential health and safety requirements set out in European Directives.

2.1 FCC Statement

This device contains FCC ID: 5123A-BGTWT32AE and complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference and (2) this device must accept any interference received, including interference that may cause undesired operation.

Modifications not expressly approved by Tobii Dynavox could void the user's authority to operate the equipment under FCC rules.

2.1.1 For Part 15B Equipment

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications.

However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

2.1.2 For Portable Devices

FCC RF Radiation Exposure Statement:

1. This Transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.
2. This equipment complies with FCC RF radiation exposure limits set forth for an uncontrolled environment. This device was tested for typical hand held operations with the device contacted directly to the human body to the sides of the device. To maintain compliance with FCC RF exposure compliance requirements, avoid direct contact to the transmitting antenna during transmitting.

2.2 Industry Canada Statement

Caution

- (i) The device for operation in the band 5150-5250 MHz is only for indoor use to reduce the potential for harmful interference to co-channel mobile satellite systems;
- (ii) The maximum antenna gain permitted for devices in the bands 5250-5350 MHz and 5470-5725 MHz shall be such that the equipment still complies with the e.i.r.p. limit;
- (iii) The maximum antenna gain permitted for devices in the band 5725-5850 MHz shall be such that the equipment still complies with the e.i.r.p. limits specified for point-to-point and non-point-to-point operation as appropriate; and
- (iv) Users should also be advised that high-power radars are allocated as primary users (i.e. priority users) of the bands 5250-5350 MHz and 5650-5850 MHz and that these radars could cause interference and/or damage to LE-LAN devices.

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This device contains IC ID: 5132A-BGTWT32AE and has been tested to the safety requirements for exposure to radio waves (SAR) in RSS-102 to the General Public (Uncontrolled Environment) limits (1.6W/Kg). The highest SAR value for the Speech Case device is 0.3 W/kg. This device can be operated safely with a minimum distance 0 cm between the equipment and the user's body.

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This Class B digital apparatus complies with Canadian ICES-003.

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This device complies with Industry Canada license-exempt RSS standard(s). Operation is subject to the following two conditions: (1) this device may not cause interference, and (2) this device must accept any interference, including interference that may cause undesired operation of the device.

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2.3 CE Statement

This equipment complies with the requirements relating to electromagnetic compatibility, the essential protection requirement of Electromagnetic Compatibility (EMC) Directive 2014/30/EU on the approximation of the laws of the Member States relating to electromagnetic compatibility and Radio Equipment Directive (RED) 2014/53/EU to meet the regulation of the radio equipment and telecommunications terminal equipment.

2.4 Directives and Standards

The Speech Case complies with the following directives:

- Low voltage Directive 2014/35/EU
- Electromagnetic Compatibility (EMC) Directive 2014/30/EU
- Radio Equipment Directive (RED) 2014/53/EU

- RoHS2 Directive 2011/65/EU
- WEEE Directive 2012/19/EU
- Reach Directive 2006/121/EC, 1907/2006/EC Annex 17
- Batteries Directive 2013/56/EU

The device has been tested to comply with EN 62368-1, ISO 14971:2007 and EN ISO 14971:2012 and other relevant standards for the intended markets.

3 Customer Support

For support, please contact your local representative or Support at Tobii Dynavox. In order to receive assistance as quickly as possible, make sure you have access to your Speech Case device and, if possible, an Internet connection. You should also be able to supply the serial number of the device, which you find on the back of the device.

For further product information and other support resources, please visit the Tobii Dynavox website www.tobiidynavox.com.

4 Disposing of the Speech Case Device

Do not dispose of the Speech Case device in general household or office waste. Follow your local regulations for the disposal of electrical and electronic equipment.

5 Disposing of the Batteries

Do not dispose of the batteries in general household or office waste. Follow your local regulations for the disposal of batteries.

6 Technical Specifications

6.1 Speech Case

Standard	Speech Case
Type	Speech accessory for 10.2" iPad (7th generation)
Model	Speech Case
Dimensions (WxHxD)	273 × 196 × 52 mm 10.7 × 7.7 × 2.0 inches
Weight	624 g 1 lb 6 oz
Speakers	2 × 31 mm × 9 mm, 4.0 ohms, 5 W
Connectors	2 × 3.5 mm Switch Jack Ports 1 × USB-C
Buttons	1 × Volume Up 1 × Volume Down 1 × 3 Position Selector
Bluetooth®	Bluetooth® 3.0
Battery Run Time	~ 8 hours. normal usage
Battery Technology	Li-ion Polymer rechargeable battery
Battery Capacity	15.54 Wh
Battery Charge Time	~ 4 hours
Power Supply	5.0 VDC 4.8A AC adapter

6.2 Power Adapter

Item	Specification
Model	SPAD92BC-20
Input voltage	100 to 240 VAC
Input current (max)	0.48 A
Input frequency	50 to 60 Hz
Output current	4.8A
Rated output voltage	5 VDC

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