

## Release Notes **Tobii Dynavox Update Notifier**

### Introduction

The Tobii Dynavox Update Notifier is a program for automatic update of selected Tobii Dynavox applications for the following platforms:

- C-Series
- I-Series
- I-Series+
- PCEye
- PCEye Go
- PCEye Explore

The Update Notifier can be activated to run automatically on a schedule or triggered manually.

### System Requirements

- Windows Vista (SP2) 7, 8, 8.1 or 10
- .NET Framework 4 Full
- I-Series, C-Series or any PCEye device

**Customer support:** For more information or support, please visit [www.TobiiDynavox.com](http://www.TobiiDynavox.com).

### Release Version 1.4.2

**Release date:** 2016-01-11

#### Improvements

- Download suggestions of licensed products are based on the user's currently installed license (POP-100)
- Included MD5 checksum verification for downloaded installers (TITI-234)

### Release Version 1.4.1

**Release date:** 2015-11-05

#### Improvements

- The close applications dialog aligns with the window style of the given operating system (POP-101)
- Added prevention for double auto start items for Update Notifier (POP-102, POP-103)
- Fixed a bug that made it possible to start more than one instance of Update Notifier (POP-105)

### Release Version 1.4.0

**Release date:** 2015-09-15

#### New Features

- Support for Windows 10 (POP-97)

### Support for Your Tobii Dynavox Device

#### Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: [www.TobiiDynavox.com](http://www.TobiiDynavox.com) or [www.myTobiiDynavox.com](http://www.myTobiiDynavox.com).

#### Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit [www.TobiiDynavox.com/contact](http://www.TobiiDynavox.com/contact)

## Improvements

- The confirmation dialog aligns with the window style of the given operating system (POP-99)

## Release Version 1.3.3

**Release date:** 2015-07-10

## Improvements

- Improved logging for Update Notifier (POP-96)
- Fixed bug where updating from TGIS 2.6.3 or earlier to 2.6.4 or later would fail due to naming change (TAM-2096)
- Fixed error when a connection failure dialog is shown due to missing state file (TITI-387)

## Release Version 1.3.2

**Release date:** 2015-06-08

## New Features

- Support for PCEye Explore eye tracker (TGP-278)
- Support for I-Series+ device (TITI-153)
- Change of name to Tobii Dynavox Update Notifier (TGP-315)

## Improvements

- Identification of I-Series devices done through serial number instead of depending on values in the registry (TAM-1805)
- Fixed a bug where I-Series devices would show an incorrect state of the automatic update settings after the deployment finished (TITI-129)
- Fixed a bug where new Update Notifier windows was opened indefinitely, if the application was started by a user account without administrative rights (TITI-133)
- Fixed a bug where the computer could go into sleep mode while Update Notifier was updating, causing the update to fail (TITI-196)
- Fixed a bug where the application could close before the installations are complete (TITI-236)
- Fixed a bug where the list of available updates could be scrolled slightly in horizontal direction (TITI-285)
- Fixed a bug where the application could crash if the internet connection was very shaky while connecting to the server (TITI-287)

## Release Version 1.3.1

**Release date:** 2014-06-30

## Improvements

- Fix issue where non-executable file was set to start with windows (NESDK-1756)

## Release Version 1.2.0

**Release date:** 2013-05-28

## Support for Your Tobii Dynavox Device

### Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: [www.TobiiDynavox.com](http://www.TobiiDynavox.com) or [www.myTobiiDynavox.com](http://www.myTobiiDynavox.com).

### Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit [www.TobiiDynavox.com/contact](http://www.TobiiDynavox.com/contact)

## New Features

- Support for Tobii I-Series (TAM-63)

## Improvements

- Reduced probability of having to run Update Notifier more than once to get completely updated system (TAM-479)
- Updated icon (TAM-753)
- Different behavior when using minimize or close window (TAM-887)
- Update Notifier does not cancel started downloads in Windows if the program is closed before the download is complete (TAM-1394)

## Support for Your Tobii Dynavox Device

### Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: [www.TobiiDynavox.com](http://www.TobiiDynavox.com) or [www.myTobiiDynavox.com](http://www.myTobiiDynavox.com).

### Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit [www.TobiiDynavox.com/contact](http://www.TobiiDynavox.com/contact)