

Release Notes **Tobii Dynavox Windows Control**

Introduction

Tobii Dynavox Windows Control is a software solution for controlling Windows applications using gaze with Tobii Dynavox eye trackers.

System Requirements

- Windows 7, 8.1, or 10
- .NET Framework 4.5

Customer support: For more information or support, please visit www.TobiiDynavox.com.

Release Version 2.0.1

Release date: 2017-07-10

New Features

- Support for EyeMobile Plus (TGP-2084, TGP-2373)

Improvements

- Installers installed by the Windows Control bundle can now be repaired through Windows (TGP-1666)
- Made it possible to switch to sticky left click using mouse (TGP-1929)
- Corrected Windows Control Configuration Guide welcome page button's font (TGP-2032)
- Fixed inconsistent highlighting in the manage profile view in Settings (TGP-2099)
- Fixed a bug where the incorrect calibration type was shown if the serial number of the eye tracker could not be retrieved (TGP-2113, TGP-2210)
- Improved look and feel of the track status window in it's smallest size (TGP-2146)
- Fixed a bug where the Shortcuts window would close when pausing Windows Control from the Controller window (TGP-2166)
- Fixed an incorrect Portuguese translation for the eye tracking disabled notification (TGP-2183)
- Added pause and resume to the tray icon's context menu (TGP-2193)
- Fixed a bug where the "More Info" button in Settings was not possible to click using mouse when gaze interaction was paused (TGP-2196)
- Fixed a bug where the feedback dot would show in the Windows Control zoom window when using secondary sticky left click (TGP-2200)
- Fixed an error in the Windows Control shutdown process (TGP-2201)
- Fixed a bug where the Controller window could end up outside of all screens when changing eye tracked screen (TGP-2204)
- Fixed a typo, "Wlndows", for Norwegian (TGP-2205)
- Fixed a bug with the distance meter of the track status when using PCEye Mini (TGP-2221)
- Updated the bundled version of Tobii Eye Tracking to 2.8.2 (TGP-2230)
- Renamed the 'Next' button to 'Start' in the Calibration Name Profile (TGP-2240)
- Fixed a bug where regions with negative dwell times could crash Region Interaction (TGP-2244)
- Fixed a bug where the Windows Control taskbar could become mispositioned if another appbar is docked to top (TGP-2249)
- Updated font on the Controller window's button (TGP-2264)

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com or www.myTobiiDynavox.com.

Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact

- Fixed incorrect Chinese translation of "Activation Method" (TGP-2265)
- Fixed a bug where the Windows Control keyboard would get stuck on the screen when locking and unlocking Windows while the keyboard was open (TGP-2290)
- Fixed a bug where the system information file would not be created if any of its intended content could not be accessed (TGP-2307)
- Fixed cut off text in the notification for Tobii Service not running (TGP-2309)
- Fixed a deadlock in Region Interaction when opening and closing the Windows Control keyboard from other applications (TGP-2310)
- Fixed a bug where there could be a lingering feedback dot after turning Windows Control off (TGP-2316)
- Changed the default dwell time to 1100 milliseconds (TGP-2331)
- Updated the taskbar task selection UI in Settings to better fit translated texts (TGP-2337)
- Updated the tutorial UI to better fit translated texts (TGP-2350)
- Made the Test Eye Tracking view topmost (TGP-2355)
- Fixed inconsistent capitalization of text on the tiles of the Setting application's start page (TGP-2357)
- Fixed a bug in the German Configuration Guide where "Konfigurationsassistent" would split to two lines (TGP-2393)
- Updated the bundled version of Virtual Remote to 1.0.1 (TGP-2417)
- Updated the bundled version of Update Notifier to 1.5.1 (TGP-2444)

Copyright ©Tobii AB (publ). Illustrations and specifications do not necessarily apply to products and services offered in each local market. Technical specifications are subject to change without prior notice. All other trademarks are the property of their respective owners.

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com or www.myTobiiDynavox.com.

Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact