


Communicator 5

Connecting Phone to device:

1. Turn on Bluetooth on your cell phone and make sure it is visible to other devices.
2. Open the **quick menu**
(touch and hold anywhere on the screen or press this button on the left side of the device. You can also right click if you are using a mouse. Or select Ctrl +M on the keyboard)

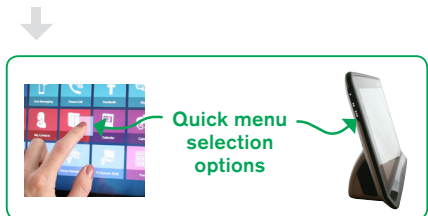

3. Select **Caregiver Settings**.
4. Select **Mobile Phone** (make sure Bluetooth is enabled in Mobile Phone Settings)

5. If Bluetooth is not enabled, open the **Control Center** and select **Enable Wireless** units check box. Close the menu when finished.
6. Select the **Mobile Phone** button
(Communicator will search for any devices within range).
7. Select your phone from the list and then select the **Connect** button.
8. Make sure the pairing codes are the same on both your device screen and your phone
9. Your phone may prompt you to allow Communicator to access your messages. Press **allow** and then **OK** on the device.

For more information watch the video at

Watch dynavotech.force.com/devices/apex/Communicator

Please note you will need a mytobiidynavox.com account to access the resources

